

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No.66074 Part Number: HCV10045

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ALPHA HCV by PCR TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 10/28/1999	ABBOTT ANTI-HCV 3.0 Lab # 5 TEST DATE: 10/16/2000
10045-01	01/12/98	13	POSITIVE	0.178	0.540	0.412	0.330
10045-02	01/14/98	8	POSITIVE	0.176	0.474	0.392	0.250
10045-03	01/20/98	43	POSITIVE	0.151	0.415	0.431	0.330
10045-04	01/22/98	45	POSITIVE	0.152	0.463	0.373	0.290
10045-05	01/27/98	88	POSITIVE	0.122	0.443	0.314	0.170

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5 TEST DATE: 10/17/2000	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO Enhanced SAVE ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/29/2000	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
10045-01	0.280	0.090	0.118	0.010	0.07	0.02	0.08
10045-02	0.240	0.070	0.084	0.010	0.07	0.02	0.08
10045-03	0.290	0.060	0.105	0.010	0.07	0.02	0.09
10045-04	0.270	0.060	0.098	0.000	0.07	0.02	0.08
10045-05	0.240	0.060	0.098	0.000	0.06	0.02	0.08

#### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10045 Rev04  
Effective Date: 10/14/2021