

ZeptoMetrix®

HCV Seroconversion Panel Donor No.67861 Part Number: HCV10052

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype:3a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Architect ANTI-HIV	ABBOTT PRISM ANTI-HCV	ABBOTT ANTI-HCV Lab # 1	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5	ORTHO VITROS ECi ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/19/1999	TEST DATE: 10/4/2000
10052-01	04/27/98	14	NEGATIVE	0.130	0.317	0.130	0.166	0.154	0.000	0.05
10052-02	04/29/98	8	NEGATIVE	0.131	0.331	0.110	0.135	0.154	0.000	0.05
10052-03	05/25/98	15	NEGATIVE	0.119	0.307	0.100	0.124	0.154	0.000	0.04
10052-04	05/27/98	7	NEGATIVE	0.101	0.279	0.080	0.101	0.154	0.000	0.04
10052-05	06/01/98	13	NEGATIVE	0.109	0.293	0.090	0.160	0.128	0.000	0.04
10052-06	06/04/98	12	NEGATIVE	0.104	0.286	0.080	0.147	0.128	0.000	0.03
10052-07	06/08/98	17	NEGATIVE	0.101	0.303	0.070	0.115	0.128	0.000	0.03
10052-08	06/11/98	13	NEGATIVE	0.095	0.310	0.070	0.125	0.103	0.000	0.03
10052-09	07/10/98	15	POSITIVE	0.118	0.613	0.090	0.137	0.154	0.000	0.04
10052-10	07/12/98	15	POSITIVE	0.118	0.331	0.070	0.117	0.128	0.000	0.04
10052-11	07/27/98	16	POSITIVE	0.115	0.383	0.120	0.677	0.154	0.000	0.04
10052-12	07/29/98	13	POSITIVE	0.101	0.512	0.100	0.295	0.128	0.000	0.04

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10052 Rev04
Effective Date: 10/14/2021