

ZeptoMetrix®

HCV Seroconversion Panel Donor No.490104584 Part Number: HCV10062

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	EY LABS HCVSCAN
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/23/1999
10062-01	09/27/96	8	0.116	0.318	0.220	0.090	NEGATIVE
10062-02	09/30/96	4	0.107	0.315	0.170	0.097	NEGATIVE
10062-03	10/04/96	7	0.110	0.428	0.200	0.105	NEGATIVE
10062-04	10/07/96	8	0.104	11.538	0.180	0.096	NEGATIVE
10062-05	10/11/96	12	0.111	9.908	0.200	0.105	NEGATIVE
10062-06	10/14/96	16	0.125	10.829	0.180	0.092	NEGATIVE
10062-07	11/07/96	269	5.000	11.791	3.900	1.961	POSITIVE
10062-08	11/11/96	259	5.288	12.171	5.950	2.800	POSITIVE

PANEL MEMBER	CHIRON PCR	VENDOR ANTI-HCV Ab	ORTHO HCV Ab	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10062-01	<0.2	NEGATIVE	0.046	0.05	0.01	0.10
10062-02	<0.2	NEGATIVE	0.061	0.05	0.01	0.10
10062-03	0.229	NEGATIVE	0.046	0.06	0.01	0.11
10062-04	114.1	NEGATIVE	0.046	0.06	0.01	0.11
10062-05	77.49	NEGATIVE	0.426	0.06	0.01	0.10
10062-06	69.63	NEGATIVE	0.046	0.05	0.01	0.10
10062-07	50.25	POSITIVE	2.040	0.06	9.63	0.11
10062-08	57.03	POSITIVE	5.023	0.07	17.3	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10062 Rev03
Effective Date: 10/15/2021