

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No.72237 Part Number: HCV10165

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ALPHA HCV PCR TEST DATE: UNK	EY LABS HCVSCAN TEST DATE: 7/6/2000	VENDOR ANTI-HCV Ab TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT HCV Ab TEST DATE: 6/27/2000	ABBOTT Architect ANTI-HCV TEST DATE: UNK
10165-01	06/21/99	24	POSITIVE	NEGATIVE	NEGATIVE	0.148	8.955	0.158	0.070
10165-02	06/23/99	21	POSITIVE	NEGATIVE	NEGATIVE	0.195	11.324	0.158	0.080
10165-03	07/03/99	18	POSITIVE	NEGATIVE	NEGATIVE	0.178	11.157	0.158	0.080
10165-04	07/05/99	17	POSITIVE	NEGATIVE	NEGATIVE	0.130	8.969	0.158	0.060
10165-05	07/10/99	48	POSITIVE	POSITIVE	NEGATIVE	0.496	7.293	0.237	0.180
10165-06	07/15/99	96	POSITIVE	POSITIVE	NEGATIVE	5.327	9.003	2.184	5.950
10165-07	07/17/99	104	POSITIVE	POSITIVE	NEGATIVE	5.329	12.216	3.447	7.720
10165-08	07/22/99	140	POSITIVE	POSITIVE	NEGATIVE	5.395	11.916	5.263	8.080
10165-09	07/24/99	200	POSITIVE	POSITIVE	NEGATIVE	5.336	11.519	5.263	9.390

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO HCV Ab TEST DATE: 6/27/2000	ORTHO RIBA 5-1-1 TEST DATE: 7/6/2000	ORTHO RIBA C33 TEST DATE: 7/6/2000	ORTHO RIBA C22 TEST DATE: 7/6/2000	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
10165-01	0.092	0.135	NEGATIVE	POSITIVE/NEGATIVE	NEGATIVE	0.06	0.35	0.09
10165-02	0.084	0.195	NEGATIVE	POSITIVE/NEGATIVE	NEGATIVE	0.06	0.21	0.09
10165-03	0.091	0.135	NEGATIVE	POSITIVE/NEGATIVE	NEGATIVE	0.06	0.31	0.10
10165-04	0.090	0.135	NEGATIVE	POSITIVE/NEGATIVE	NEGATIVE	0.07	0.30	0.10
10165-05	0.104	0.150	NEGATIVE	POSITIVE/NEGATIVE	2+	0.06	1.44	0.08
10165-06	1.571	2.129	NEGATIVE	POSITIVE	3+	0.06	13.5	0.09
10165-07	2.712	2.789	NEGATIVE	POSITIVE	3+	0.05	15.2	0.10
10165-08	4.504	2.774	POSITIVE	2+	3+	0.06	20.3	0.11
10165-09	5.297	4.798	1+	2+	3+	0.07	21.7	0.10

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10165 Rev03  
Effective Date: 10/15/2021