

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No.70225 Part Number: HCV10185

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV PCR	VENDOR ANTI-HCV Ab	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT HCV Ab	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/6/2000	TEST DATE: UNK
10185-01	02/18/99	3	NEGATIVE	NEGATIVE	0.103	0.338	0.145	0.050
10185-02	06/28/99	66	POSITIVE	POSITIVE	5.192	3.139	2.727	9.800
10185-03	07/02/99	236	POSITIVE	POSITIVE	5.252	3.094	>3.363	11.050
10185-04	07/08/99	43	NEGATIVE	POSITIVE	5.222	4.683	>3.636	10.300
10185-05	07/12/99	132	POSITIVE	POSITIVE	5.225	5.073	>3.636	11.850

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ORTHO HCV Ab	TMA TEST	ORTHO RIBA NS5	ORTHO RIBA SOD	ORTHO RIBA Interpretation
	TEST DATE: UNK	TEST DATE: 6/6/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10185-01	0.085	0.018	0.18	POSITIVE	NEGATIVE	INDETERMINATE
10185-02	7.013	5.195	10.54	POSITIVE	NEGATIVE	POSITIVE
10185-03	7.623	5.195	9.66	POSITIVE/NEGATIVE	NEGATIVE	INDETERMINATE
10185-04	7.647	5.195	10.01	2+	NEGATIVE	POSITIVE
10185-05	7.985	5.195	10.24	2+	NEGATIVE	POSITIVE

PANEL MEMBER	ORTHO RIBA C100(NS4)	ORTHO RIBA C33(NS3)	ORTHO RIBA C22 CORE	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10185-01	POSITIVE/NEGATIVE	NEGATIVE	NEGATIVE	0.08	0.01	0.10
10185-02	POSITIVE/NEGATIVE	4+	NEGATIVE	0.09	10.1	0.11
10185-03	POSITIVE/NEGATIVE	4+	NEGATIVE	0.09	12.3	0.12
10185-04	POSITIVE/NEGATIVE	4+	NEGATIVE	0.10	12.8	0.11
10185-05	POSITIVE/NEGATIVE	4+	NEGATIVE	0.10	12.5	0.10

#### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10185 Rev03  
Effective Date: 10/15/2021