



**HCV Seroconversion Panel Donor No. 61025
Part Number: HCV6217**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT IMx ANTI-HCV TEST DATE: 9/23/1996	ABBOTT 2.0 ANTI-HCV TEST DATE: UNK	ABBOTT Architect ANTI-HCV TEST DATE: UNK
6217-01	01/31/96	7	0.129	0.351	0.494	0.404	0.160
6217-02	02/05/96	15	0.132	0.344	0.515	0.468	0.150
6217-03	02/07/96	8	0.138	0.321	0.532	0.474	0.170
6217-04	02/12/96	16	0.141	0.525	0.494	0.608	0.170
6217-05	02/18/96	14	0.153	0.318	0.508	1.108	0.160

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO ANTI-HCV 3.0 (SAVe) TEST DATE: 8/15/1996	ORTHO ANTI-HCV 3.0 TEST DATE: 8/6/1996	ORTHO VITROS Eci ANTI-HCV TEST DATE: UNK	SANOFI Monolisa ANTI-HCV New Antigens TEST DATE: 9/25/1996	ORTHO RIBA 3.0 C100 TEST DATE: UNK	ORTHO RIBA 3.0 C33 TEST DATE: UNK
6217-01	0.240	0.119	0.038	0.09	0.205	+/-	-
6217-02	0.139	0.146	0.058	0.06	0.221	+/-	-
6217-03	0.183	0.125	0.035	0.09	0.240	+/-	-
6217-04	0.163	0.331	0.093	0.15	0.371	2+	+/-
6217-05	0.171	0.767	0.215	0.32	0.605	2+	-

PANEL MEMBER	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK	ORTHO RIBA 3.0 SOD TEST DATE: UNK	ORTHO RIBA 3.0 TEST DATE: UNK	CHIRON HCV RNA bDNA (copies/mL) TEST DATE: UNK	PCR RNA Qualitative Result Lab # 3 TEST DATE: 9/13/1996	PCR RNA Quantitative Result Lab # 3 TEST DATE: 9/13/1996
6217-01	-	-	-	-	<200,000	NEGATIVE	<100
6217-02	-	-	-	-	<200,000	NEGATIVE	<100
6217-03	-	-	-	-	<200,000	NEGATIVE	<100
6217-04	-	-	+/-	INDETERMINATE	<200,000	NEGATIVE	<100
6217-05	-	-	-	INDETERMINATE	<200,000	NEGATIVE	<100

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV6217 Rev03
Effective Date: 10/07/2021