



HCV Seroconversion Panel Donor No. 60749
Part Number: HCV6219

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT IMx ANTI-HCV	ABBOTT ANTI-HCV 2.0	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/30/1996	TEST DATE: 9/6/1996	TEST DATE: UNK	TEST DATE: UNK
6219-01	10/19/95	10	0.136	0.344	0.000	0.374	0.410	0.381
6219-02	10/24/95	4	0.153	0.375	0.000	0.392	0.470	0.510
6219-03	10/31/95	12	0.145	0.355	0.000	1.240	1.620	1.167
6219-04	11/02/95	12	0.129	0.334	0.000	1.398	1.600	1.588
6219-05	11/07/95	10	0.139	0.338	0.000	1.573	1.990	1.654
6219-06	11/09/95	14	0.126	0.314	0.000	1.544	2.040	1.663
6219-07	11/14/95	8	0.139	0.351	0.438	2.058	2.750	2.846
6219-08	11/16/95	7	0.130	0.321	0.462	2.263	2.660	2.282
6219-09	11/21/95	8	0.148	0.341	0.470	2.100	3.160	2.885
6219-10	11/28/95	8	0.177	0.368	0.466	2.187	3.030	2.445
6219-11	12/06/95	12	0.174	0.341	0.472	2.299	3.170	2.189
6219-12	12/08/95	7	0.147	0.351	0.515	2.287	3.150	2.595
6219-13	12/13/95	2	0.148	0.324	0.509	2.208	3.010	2.453
6219-14	12/17/95	10	0.145	0.324	0.603	2.015	2.990	2.542

PANEL MEMBER	ORTHO ANTI-HCV 3.0 (SAVe)	ORTHO ANTI-HCV 3.0	SANOFI Monolisa ANTI-HCV New Antigens	ORTHO RIBA 3.0 C100	ORTHO RIBA 3.0 C33	ORTHO RIBA 3.0 C22	ORTHO RIBA 3.0 NS5
	TEST DATE: 8/15/1996	TEST DATE: UNK	TEST DATE: 9/25/1996	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6219-01	0.256	0.041	0.970	-	+/-	-	-
6219-02	0.330	0.054	0.876	-	+/-	-	-
6219-03	1.887	0.355	3.024	-	1+	-	-
6219-04	1.842	0.493	3.567	-	1+	-	-
6219-05	2.812	0.683	4.353	-	2+	-	-
6219-06	2.095	0.490	3.843	-	2+	-	-
6219-07	2.917	0.809	5.168	-	2+	-	-
6219-08	3.345	0.789	5.118	-	2+	-	-
6219-09	3.405	0.949	5.310	-	2+	-	-
6219-10	2.938	1.663	5.371	-	2+	-	-
6219-11	3.527	1.542	5.199	-	2+	-	-
6219-12	2.815	1.471	5.259	-	2+	-	-
6219-13	3.253	1.463	5.384	-	2+	-	-
6219-14	2.592	1.218	4.575	-	2+	-	-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



**HCV Seroconversion Panel Donor No. 60749
Part Number: HCV6219**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO RIBA 3.0 SOD TEST DATE: UNK	CHIRON HCV RNA bDNA (copies/mL) TEST DATE: 12/17/1996	PCR RNA Qualitative Result Lab # 3 TEST DATE: 9/13/1996	PCR RNA Quantitative Result Lab # 3 TEST DATE: 9/13/1996	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
6219-01	-	<200000	NEGATIVE	<100	4.26	0.11	0.10
6219-02	-	<200000	NEGATIVE	<100	1.76	0.09	0.09
6219-03	-	<200000	NEGATIVE	<100	0.34	0.45	0.10
6219-04	-	<200000	NEGATIVE	<100	1.30	0.58	0.10
6219-05	-	<200000	NEGATIVE	<100	0.62	0.88	0.09
6219-06	-	<200000	NEGATIVE	<100	0.28	0.75	0.09
6219-07	-	<200000	NEGATIVE	<100	0.21	1.06	0.09
6219-08	-	<200000	NEGATIVE	<100	0.09	1.12	0.09
6219-09	-	<200000	NEGATIVE	<100	0.05	1.21	0.10
6219-10	-	<200000	NEGATIVE	<100	0.20	1.51	0.11
6219-11	-	<200000	NEGATIVE	<100	0.07	1.67	0.10
6219-12	-	<200000	NEGATIVE	<100	0.06	1.39	0.10
6219-13	-	<200000	NEGATIVE	<100	1.16	1.48	0.10
6219-14	-	216000	NEGATIVE	<100	0.11	1.30	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com