

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 62680 Part Number: HCV6224

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT IMx ANTI-HBc TEST DATE: 6/18/1997	ABBOTT ANTI-HCV 2.0 Lab # 1 TEST DATE: 8/14/1997	ABBOTT ANTI-HCV 2.0 Lab # 2 TEST DATE: 6/25/1997	ABBOTT AxSYM ANTI-HCV TEST DATE: 6/18/1997
6224-01	10/28/96	13	0.135	1.565	0.46	0.22	0.14	0.48
6224-02	10/31/96	22	0.139	1.732	0.43	0.22	0.15	0.50
6224-03	11/04/96	61	0.130	1.388	0.44	0.23	0.18	0.49
6224-04	11/08/96	75	0.191	0.866	0.49	0.23	0.19	0.51
6224-05	11/16/96	73	1.782	4.137	0.54	0.27	0.16	0.62
6224-06	11/19/96	61	2.977	4.599	0.54	0.23	0.17	0.63

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: 6/18/1997	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO ANTI-HCV 3.0 (SAVe) TEST DATE: UNK	ORTHO ELISA ANTI-HCV 3.0 TEST DATE: 6/17/1997	BOEHRINGER MANNHEIM Ezymuntest ANTI-HCV TEST DATE: 6/5/1997	ORTHO RIBA 3.0 C-100 TEST DATE: UNK	ORTHO RIBA 3.0 C33 TEST DATE: UNK
6224-01	0.08	0.090	0.080	0.005	0.06	0.01	-	-
6224-02	0.06	0.080	0.080	0.007	0.05	0.01	-	-
6224-03	0.06	0.090	0.080	0.010	0.05	0.01	-	-
6224-04	0.12	0.230	0.130	0.008	0.07	0.01	-	+/-
6224-05	1.44	2.630	1.440	1.063	0.54	0.02	-	1
6224-06	2.35	3.520	2.500	2.436	0.97	0.03	-	2

PANEL MEMBER	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK	ORTHO RIBA 3.0 SOD TEST DATE: UNK	PCR RNA Qualitative Result Lab # 3 TEST DATE: UNK	PCR RNA Quantitative Result Lab # 4 TEST DATE: UNK	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
6224-01	-	-	-	POSITIVE	230000	0.06	0.01	0.11
6224-02	-	-	-	POSITIVE	240000	0.06	0.01	0.13
6224-03	-	-	-	POSITIVE	150000	0.07	0.01	0.14
6224-04	-	-	-	POSITIVE	86000	0.07	0.03	0.12
6224-05	-	-	-	POSITIVE	270000	0.07	0.96	0.11
6224-06	-	-	-	POSITIVE	310000	0.06	1.58	0.11

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6224 Rev03  
Effective Date: 06/15/2021