

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 62804 Part Number: HCV6229

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Corzyme ANTI-HBc	ABBOTT ANTI-HCV 2.0 Lab # 1	ABBOTT ANTI-HCV 2.0 Lab # 2	ABBOTT Architect ANTI-HCV	ORTHO ANTI-HCV 3.0 (SAVe)
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/6/1997	TEST DATE: UNK	TEST DATE: 10/14/1997
6229-01	11/14/96	4	0.141	9.443	2.09	0.387	0.35	0.380	0.03
6229-02	11/17/96	4	0.129	9.530	1.92	0.414	0.34	0.340	0.02
6229-03	11/21/96	9	0.133	5.318	1.91	0.349	0.34	0.320	0.02
6229-04	11/24/96	62	0.154	10.696	1.86	0.389	0.33	0.390	0.02
6229-05	12/01/96	77	1.014	6.007	1.94	0.477	0.39	1.790	0.53
6229-06	12/04/96	38	1.596	8.953	2.05	0.466	0.38	2.430	0.87
6229-07	12/08/96	78	2.743	10.037	1.98	0.534	0.48	4.610	1.71
6229-08	12/12/96	148	5.295	11.209	1.86	1.084	1.01	8.460	3.98

PANEL MEMBER	ORTHO RIBA 2.0	ORTHO RIBA 3.0 C100	ORTHO RIBA 3.0 C33	ORTHO RIBA 3.0 C22	ORTHO RIBA 3.0 NS5	ORTHO RIBA 3.0 SOD	ORTHO RIBA 3.0	ORTHO ELISA Anti-HCV 3.0
	TEST DATE: 10/3/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/6/1997
6229-01	-	-	-	-	-	-	-	0.010
6229-02	-	-	-	-	-	-	-	0.008
6229-03	-	-	-	-	-	-	-	0.011
6229-04	-	-	-	-	-	-	-	0.011
6229-05	-	-	-	-	-	-	-	0.528
6229-06	-	-	+/-	-	-	-	-	1.064
6229-07	-	-	1+	+/-	-	-	INDETERMINATE	1.705
6229-08	-	-	3+	+/-	+/-	-	INDETERMINATE	4.072

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ROCHE DIAGNOSTICS Cobas ANTI-HCV	ABBOTT AxSYM ANTI-HCV	CHIRON bDNA (MEq/mL)	ROCHE HCV MONITOR (copies/mL)	PCR RNA Quantitative Result Lab # 3	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: 11/7/1997	TEST DATE: 10/21/1997	TEST DATE: 9/25/1997	TEST DATE: UNK	TEST DATE: 10/25/1997	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
6229-01	0.520	0.17	0.55	52.19	830000	>5000000	0.06	0.02	0.13
6229-02	0.540	0.14	0.54	55.83	710000	>5000000	0.07	0.02	0.13
6229-03	0.470	0.18	0.51	55.53	650000	>5000000	0.06	0.02	0.12
6229-04	0.540	0.16	0.53	94.76	1600000	>5000000	0.07	0.03	0.13
6229-05	1.640	0.21	0.89	31.88	520000	>5000000	0.06	0.54	0.13
6229-06	2.180	0.38	1.37	69.85	920000	>5000000	0.07	1.21	0.12
6229-07	3.460	0.67	1.50	51.02	1100000	>5000000	0.08	2.60	0.12
6229-08	5.790	3.4	3.69	43.89	710000	>5000000	0.08	8.32	0.13

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6229 Rev03  
Effective Date: 10/08/2021