



HCV Seroconversion Panel Donor No. 63625
Part Number: HCV9041

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	EY LABS HCV SCAN TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2 TEST DATE: 6/15/19987	ABBOTT ANTI-HCV 3.0 TEST DATE: 7/8/1998	ABBOTT Architect ANTI-HCV TEST DATE: UNK
9041-01	02/07/97	38	0.110	0.361	NEGATIVE	0.219	0.173	0.12	0.070
9041-02	03/03/97	31	0.113	1.990	NEGATIVE	0.238	0.182	0.11	0.090
9041-03	03/06/97	20	0.109	8.861	NEGATIVE	0.247	0.191	0.12	0.070
9041-04	03/10/97	11	0.123	10.645	NEGATIVE	0.104	0.175	0.15	0.070
9041-05	04/10/97	192	5.186	11.828	POSITIVE	2.490	3.843	MAX.ABS.	10.770
9041-06	04/12/97	276	5.138	11.838	POSITIVE	4.242	3.784	MAX.ABS.	12.070
9041-07	04/17/97	140	5.203	11.351	POSITIVE	3.723	4.556	MAX.ABS.	12.960
9041-08	04/19/97	152	5.186	11.787	POSITIVE	>4.329	4.556	MAX.ABS.	13.830

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 7/6/1998	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 8/4/1998	ABBOTT PRISM HCV TEST DATE: 8/5/1998	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA TEST DATE: 7/7/1998	ORTHO Anti-HCV 3.0 TEST DATE: UNK	ORTHO Enhanced SAVE ANTI-HCV 3.0 Lab # 4 TEST DATE: 6/29/1998	ORTHO Enhanced SAVE ANTI-HCV 3.0 Lab # 7 TEST DATE: 6/23/1998
9041-01	0.080	0.13	0.13	0.09	0.2	0.016	0.00	0.004
9041-02	0.100	0.20	0.13	0.12	0.3	0.013	0.00	0.006
9041-03	0.090	0.14	0.16	0.08	0.3	0.016	0.01	0.007
9041-04	0.080	0.13	0.18	0.08	10.7	0.011	0.00	0.013
9041-05	7.260	15.99	8.91	5.34	10.5	4.092	8.26	7.296
9041-06	8.100	17.80	12.10	5.87	10.7	4.092	MAX.ABS.	8.029
9041-07	8.270	29.54	20.06	5.54	10.7	4.092	MAX.ABS.	8.201
9041-08	8.550	32.42	21.80	6.12	10.7	4.092	MAX.ABS.	8.777

PANEL MEMBER	SANOFI Monolisa ANTI-HCV PLUS Lab # 4 TEST DATE: 6/30/1998	SANOFI Monolisa ANTI-HCV PLUS Lab #7 TEST DATE: 6/23/1998	ORTHO Chiron RIBA HCV 2.0 5-1-1 (NS4) TEST DATE: 7/7/1998	ORTHO Chiron RIBA HCV 2.0 C100 (NS4) TEST DATE: 7/7/1998	ORTHO Chiron RIBA HCV 2.0 C33 (NS3) TEST DATE: 7/7/1998	ORTHO Chiron RIBA HCV 2.0 C22 Core TEST DATE: 7/7/1998	ORTHO Chiron RIBA HCV 2.0 SOD TEST DATE: 7/7/1998	ORTHO Chiron RIBA HCV 2.0 Interpretation TEST DATE: 7/7/1998
9041-01	0.26	0.086	-	+/-	+/-	-	-	NEGATIVE
9041-02	0.33	0.046	-	-	-	-	-	NEGATIVE
9041-03	0.34	0.049	-	-	-	-	-	NEGATIVE
9041-04	0.32	0.044	-	+/-	+/-	-	-	NEGATIVE
9041-05	6.58	5.679	+/-	+/-	4+	2+	-	POSITIVE
9041-06	5.92	6.066	+/-	+/-	4+	2+	-	POSITIVE
9041-07	7.69	6.156	+/-	2+	4+	3+	-	POSITIVE
9041-08	7.91	6.256	+/-	2+	4+	3+	-	POSITIVE

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HCV Seroconversion Panel Donor No. 63625
Part Number: HCV9041

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c100(p)	ORTHO Chiron Riba HCV 3.0 c33c	ORTHO Chiron Riba HCV 3.0 c22(p)	ORTHO Chiron Riba HCV 3.0 NS5	ORTHO Chiron Riba HCV 3.0 Interpretation	CHIRON HCV RNA bDNA 2.0 (MEq/mL*)	ORTHO ECI HBsAg	ORTHO ECI ANTI-HCV	ORTHO ECI ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/7/1998	TEST DATE: 7/16/1998	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
9041-01	-	-	-	-	NEGATIVE	<0.2	0.06	0.01	0.15
9041-02	-	-	-	-	NEGATIVE	15.78	0.06	0.01	0.13
9041-03	-	-	-	-	NEGATIVE	63.72	0.06	0.01	0.12
9041-04	-	-	-	-	NEGATIVE	>120	0.06	0.01	0.13
9041-05	1+	4+-	+/-	+/-	POSITIVE	72.27	0.07	13.2	0.14
9041-06	1+	4+-	+/-	+/-	POSITIVE	66.27	0.06	15.0	0.12
9041-07	2+	4+-	1+	3+	POSITIVE	21.8	0.05	16.3	0.14
9041-08	3+	4+-	3+	3+	POSITIVE	11.24	0.05	17.8	0.13

www.h-h-c.com

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com