



**HCV Seroconversion Panel Donor No. 64254
Part Number: HCV9046**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2 TEST DATE: 6/15/1998	ABBOTT ANTI-HCV 3.0 TEST DATE: 7/8/1998	ORTHO ANTI-HCV 3.0 TEST DATE: UNK
9046-01	04/22/97	38	0.084	0.300	0.050	0.134	0.12	0.010
9046-02	06/30/97	180	5.323	11.464	>4.329	>4.556	MAX.ABS.	4.139
9046-03	07/03/97	244	5.359	11.642	>4.329	>4.556	MAX.ABS.	4.139
9046-04	07/07/97	60	5.350	11.648	>4.329	>4.556	MAX.ABS.	4.139
9046-05	07/10/97	36	5.317	11.703	>4.329	>4.556	MAX.ABS.	4.139

PANEL MEMBER	ABBOTT PRISM HCV TEST DATE: 8/5/1998	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 8/4/1998	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 7/8/1998	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE Cobas Core ANTI-HCV EIA TEST DATE: 7/7/1998	ORTHO Enhanced SAve HCV 3.0 Lab # 4 TEST DATE: 7/15/1998	ORTHO Enhanced SAve HCV 3.0 Lab # 7 TEST DATE: 6/25/1998
9046-01	0.09	0.18	0.18	0.070	0.070	0.6	0.18	0.003
9046-02	5.37	23.18	66.11	11.020	6.640	11.7	5.9	9.126
9046-03	5.54	25.29	64.89	11.360	6.760	11.7	MAX. ABS.	8.852
9046-04	5.83	26.23	62.03	11.190	6.820	11.7	MAX. ABS.	7.961
9046-05	5.96	22.95	59.18	10.670	6.850	11.7	MAX. ABS.	8.961

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS Lab # 4 TEST DATE: 6/30/1998	SANOFI Monolisa ANTI-HCV PLUS Lab # 7 TEST DATE: 6/25/1998	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C100(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C33(NS3) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 SOD TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: UNK
9046-01	0.10	0.062	-	-	-	-	-	-
9046-02	4.91	5.681	+/-	1+	3+	4+	-	2+
9046-03	4.46	5.331	+/-	1+	3+	4+	-	3+
9046-04	4.98	4.948	1+	1+	3+	4+	-	3+
9046-05	4.92	5.254	1+	1+	3+	4+	-	3+

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HCV Seroconversion Panel Donor No. 64254
Part Number: HCV9046**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: 7/22/1998	CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9046-01	-	-	-	NEGATIVE	<0.2	0.05	0.01	0.12
9046-02	4+	4+	-	POSITIVE	2.058	0.07	23.8	0.13
9046-03	4+	4+	-	POSITIVE	<0.2	0.06	24.4	0.13
9046-04	4+	4+	-	POSITIVE	<0.2	0.06	24.4	0.13
9046-05	4+	4+	-	POSITIVE	<0.2	0.05	24.7	0.15

www.h-h-c.com

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com