



**HCV Seroconversion Panel Donor No. 66158**  
**Part Number: HCV9050**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 6/4/1999	ABBOTT MUREX ANTI-HCV 4.0 Lab # 5 TEST DATE: 6/16/1999	ORTHO Enhanced SAVe HCV 3.0 Lab # 5 TEST DATE: 6/9/1999	SANOFI Monolisa ANTI- HCV PLUS Lab # 5 TEST DATE: 7/29/1999	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5 TEST DATE: 6/28/1999
9050-01	11/26/97	5	0.149	0.341	0.313	0.17	0.06	0.09	0.23
9050-02	12/16/97	0	0.139	0.355	0.287	0.22	0.09	0.08	0.21
9050-03	12/23/97	0	0.131	0.348	0.261	0.22	0.03	0.07	0.25
9050-04	12/28/97	1	0.115	0.331	0.287	0.25	0.03	0.07	0.20
9050-05	12/31/97	0	0.106	0.304	0.261	0.19	0.03	0.07	0.18
9050-06	01/13/98	3	0.127	0.324	0.313	0.22	0.04	0.07	0.19
9050-07	01/18/98	3	0.115	0.314	0.209	0.21	0.03	0.07	0.21
9050-08	01/23/98	2	0.122	0.334	0.235	0.28	0.04	0.06	0.16
9050-09	01/25/98	2	0.109	0.355	0.235	0.18	0.03	0.06	0.20
9050-10	01/30/98	5	0.122	0.379	0.235	0.24	0.04	0.06	0.21
9050-11	02/01/98	3	0.110	0.345	0.209	0.16	0.03	0.05	0.19
9050-12	02/17/98	0	0.389	0.375	>5.222	0.17	MAX ABS	3.49	9.06
9050-13	02/19/98	6	0.288	0.331	4.987	0.22	MAX ABS	3.06	8.42

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5 TEST DATE: 7/21/1999	ABBOTT PRISM HCV Lab # 5 TEST DATE: 6/15/1999	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	CHIRON bDNA TEST DATE: 10/27/1999	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
9050-01	0.18	0.18	0.390	0.425	<0.2	0.08	0.13	0.12
9050-02	0.19	0.20	0.340	0.316	<0.2	0.10	0.09	0.12
9050-03	0.16	0.17	0.250	0.252	<0.2	0.08	0.05	0.13
9050-04	0.15	0.14	0.250	0.252	<0.2	0.08	0.04	0.13
9050-05	0.14	0.15	0.210	0.237	<0.2	0.07	0.06	0.11
9050-06	0.16	0.19	0.320	0.319	<0.2	0.08	0.07	0.13
9050-07	0.14	0.16	0.270	0.274	<0.2	0.07	0.05	0.13
9050-08	0.14	0.16	0.260	0.245	<0.2	0.07	0.06	0.11
9050-09	0.19	0.16	0.220	0.364	<0.2	0.07	0.04	0.11
9050-10	0.12	0.15	0.240	0.241	<0.2	0.09	0.06	0.12
9050-11	0.14	0.15	0.230	0.253	<0.2	0.08	0.04	0.12
9050-12	5.04	2.50	10.620	4.138	<0.2	0.08	8.57	0.13
9050-13	4.48	2.59	10.550	4.099	<0.2	0.09	8.14	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)