



**HCV Seroconversion Panel Donor No. 66861  
Part Number: HCV9057**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ALPHA HCV by PCR (NGI)	CHIRON bDNA 2.0 (MEq/mL)	ABBOTT ANTI-HCV Lab # 1	ABBOTT MUREX ANTI-HCV 4.0
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/4/1999	TEST DATE: 10/27/1999	TEST DATE: 6/4/1999	TEST DATE: 6/16/1999
9057-01	03/13/98	10	0.118	0.314	NEGATIVE	<0.2	0.496	0.30
9057-02	03/15/98	11	0.100	0.307	NEGATIVE	<0.2	0.470	0.26
9057-03	03/23/98	13	0.132	0.290	NEGATIVE	<0.2	0.470	0.29
9057-04	03/25/98	11	0.103	0.303	NEGATIVE	<0.2	0.366	0.24
9057-05	03/30/98	12	0.100	0.314	POSITIVE	<0.2	0.392	0.27
9057-06	04/01/98	4	0.104	1.466	POSITIVE	6.54	0.366	0.16
9057-07	04/06/98	21	0.129	0.690	POSITIVE	1.75	0.366	0.24
9057-08	04/08/98	16	0.115	0.483	POSITIVE	0.88	0.392	0.23
9057-09	04/15/98	18	0.109	0.545	POSITIVE	1.70	0.392	0.20
9057-10	04/17/98	14	0.097	0.521	POSITIVE	1.18	0.339	0.23
9057-11	04/23/98	48	0.100	0.534	POSITIVE	0.95	0.392	0.17

PANEL MEMBER	ABBOTT ANTI-HCV 3.0	ABBOTT IMx HCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack	ABBOTT PRISM HCV	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	SANOFI Monalisa ANTI-HCV PLUS Lab # 5
	TEST DATE: 7/19/1999	TEST DATE: 6/23/1999	TEST DATE: 6/28/1999	TEST DATE: 6/15/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/9/1999	TEST DATE: 6/7/1999
9057-01	0.11	0.05	0.20	0.07	0.300	0.134	0.00	0.09
9057-02	0.10	0.10	0.35	0.06	0.220	0.141	0.00	0.11
9057-03	0.12	0.12	0.25	0.06	0.250	0.105	0.00	0.07
9057-04	0.11	0.08	0.24	0.06	0.230	0.089	0.00	0.06
9057-05	0.10	0.08	0.23	0.06	0.240	0.130	0.00	0.05
9057-06	0.08	0.07	0.20	0.07	0.190	0.090	0.00	0.07
9057-07	0.10	0.05	0.24	0.07	0.210	0.081	0.00	0.09
9057-08	0.09	0.09	0.17	0.06	0.200	0.088	0.00	0.06
9057-09	0.12	0.11	0.21	0.07	0.220	0.092	0.00	0.06
9057-10	0.09	0.06	0.18	0.06	0.180	0.095	0.00	0.08
9057-11	0.16	0.09	0.27	0.07	0.220	0.096	0.00	0.02

**Notes on Data:**

- This panel is for **Research Use Only. Not for use in diagnostic procedures.**
- Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
- All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
- No preservatives have been added.
- All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV9057 Rev04  
Effective Date: 10/11/2021



**HCV Seroconversion Panel Donor No. 66861  
Part Number: HCV9057**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron RIBA HCV 3.0 c100(p)/5-1-1  TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 3.0 C33  TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 3.0 c22(p)  TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 3.0 NS5  TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 2.0 Interpretation  TEST DATE: 2/3/2000	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½  TEST DATE: 11/16/2018
9058-01	-	-	-	-	-	0.06	0.01	0.14
9058-02	-	-	-	-	-	0.05	0.01	0.12
9058-03	-	-	-	-	-	0.05	0.01	0.12
9058-04	-	-	-	-	-	0.06	0.01	0.11
9058-05	-	-	-	-	-	0.06	0.01	0.11
9057-06	-	-	-	-	-	0.06	0.01	0.11
9057-07	-	-	-	-	-	0.06	0.01	0.13
9057-08	-	-	-	-	-	0.05	0.01	0.13
9057-09	-	-	-	-	-	0.05	0.01	0.13
9057-10	-	-	-	-	-	0.05	0.01	0.13
9057-11	-	-	-	-	-	0.05	0.01	0.14

www.h-h-c.com

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)