

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 67101 Part Number: HIV10234

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	VENDOR ANTI-HIV ½ TEST DATE: UNK	VENDOR HIV p24 Ag TEST DATE: 5/9/2010	ABBOTT HIV Ab 1/2 EIA TEST DATE: 5/11/1999	ABBOTT HIV Ag-1 MONOCLONAL TEST DATE: 5/11/1999
10234-01	03/30/98	NEGATIVE	NEGATIVE	0.129	0.102
10234-02	04/02/98	NEGATIVE	NEGATIVE	0.138	0.220
10234-03	05/07/98	NEGATIVE	POSITIVE	13.448	11.288

PANEL MEMBER	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/30/2018
10234-01	0.08	0.08	0.01	0.11
10234-02	0.09	0.05	0.06	0.11
10234-03	61.24	61.24	33.02	31.3

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV10234 Rev05
Effective Date: 10/26/2021