



**HIV Seroconversion Panel Donor No. 73695  
Part Number: HIV12007**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ORTHO VITROS ECi ANTI-HIV 1/2<br>TEST DATE: 8/6/2015 | GENE-PROBE PROCLEIX HIV-1/HCV ASSAY<br>TEST DATE: UNK | ROCHE COBAS AMPLICOR HIV-1 MONITOR<br>TEST DATE: UNK | VIROMED PCR<br>TEST DATE: 7/10/2000 | ABBOTT HIV-1 p24 Ag<br>TEST DATE: 6/30/2000 | ABBOTT IMx HIV 1/2 III<br>TEST DATE: 9/17/2002 | ABBOTT AxSYM HIV Ab 1/2 EIA<br>TEST DATE: 9/10/2002 | ABBOTT MUREX HIV Ab 1/2 + O (s/Co)<br>TEST DATE: 9/16/2002 |
|--------------|--------------|--|---|--|-------------------------------------|---|--|---|--|
| 12007-01     | 06/27/99     | 0.15   | NEGATIVE  | NEGATIVE   | 0                                   | 0.310                                       | 0.49   | 0.41  | 0.25   |
| 12007-02     | 06/29/99     | 0.14   | NEGATIVE  | NEGATIVE   | 0                                   | 0.293                                       | 0.43   | 0.41  | 0.22   |
| 12007-03     | 08/20/99     | 0.12   | NEGATIVE  | NEGATIVE   | 0                                   | 0.293                                       | 0.70   | 0.59  | 0.21   |
| 12007-04     | 10/22/99     | 0.3  | POSITIVE  | 316000   | 75815                               | 0.897                                       | 0.54   | 1.95  | 0.26   |
| 12007-05     | 10/24/99     | 2.97   | POSITIVE  | 118000   | 63710                               | 0.862                                       | 1.10   | 2.28  | 1.18   |
| 12007-06     | 10/29/99     | 60.1   | POSITIVE  | 54600  | 14749                               | 0.569                                       | 10.51  | 17.91   | 13.29  |
| 12007-07     | 10/31/99     | 55.3   | POSITIVE  | 33500  | 29564                               | 0.414                                       | 9.41   | 14.49   | 13.08  |
| 12007-08     | 11/05/99     | 38.7   | POSITIVE  | 300000   | 18875                               | 0.328                                       | 7.54   | 9.44  | 13.09  |
| 12007-09     | 11/07/99     | 32.8   | POSITIVE  | 26400  | 23054                               | 0.241                                       | 7.85   | 9.24  | 13.29  |

| PANEL MEMBER | ORTHO VITROS ANTI- HIV 1/2<br>TEST DATE: 8/13/2002 | TRINITY BIOTECH UNIGOLD RECOMBIGNEN HIV-1<br>TEST DATE: UNK | GENETICS SYSTEMS rLAV EIA<br>TEST DATE: 7/6/2000 | MEDMIRA ANTI-HIV-1 RAPID TEST<br>TEST DATE: UNK | ORAQUICK ADVANCE HIV 1/2 Ab RAPID TEST<br>TEST DATE: UNK | BIO-RAD HIV-1 WESTERN BLOT<br>TEST DATE: UNK |
|--------------|--|---|--|---|--|--|
| 12007-01     | 0.10   | NEGATIVE  | 0.123  | NEGATIVE  | NEGATIVE   | NOT TESTED                                   |
| 12007-02     | 0.09   | NEGATIVE  | 0.130  | NEGATIVE  | NEGATIVE   | NOT TESTED                                   |
| 12007-03     | 0.09   | NEGATIVE  | 0.120  | NEGATIVE  | NEGATIVE   | NEGATIVE                                     |
| 12007-04     | 0.17   | NEGATIVE  | 0.113  | NEGATIVE  | NEGATIVE   | NEGATIVE                                     |
| 12007-05     | 1.13   | NEGATIVE  | 0.130  | NEGATIVE  | NEGATIVE   | NEGATIVE                                     |
| 12007-06     | 30.30  | POSITIVE  | 0.933  | POSITIVE  | NEGATIVE   | P55,P40,P24,P18+/-                           |
| 12007-07     | 27.40  | POSITIVE  | 1.303  | POSITIVE  | NEGATIVE   | GP160+/-,P55,P40,P24,P18+/-                  |
| 12007-08     | 20.90  | POSITIVE  | 1.655  | POSITIVE  | POSITIVE   | GP160+/-,P65,P55,GP41+/-,P40,P31+/-,P24,P18  |
| 12007-09     | 21.20  | POSITIVE  | 1.796  | POSITIVE  | POSITIVE   | GP160+/-,P65,P55,GP41+/-,P40,P31+/-,P24,P18  |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

# ZeptoMetrix®

## HIV Seroconversion Panel Donor No. 73695 Part Number: HIV12007

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ABBOTT PRISM HIV Ag/Ab COMBO<br>TEST DATE: UNK | ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO<br>TEST DATE: UNK | ABBOTT ARCHITECT HIV Ag/Ab COMBO<br>TEST DATE: UNK | BIO-RAD MULTISPOT HIV 1/2 RAPID TEST<br>TEST DATE: UNK | BIO-RAD MULTISPOT HIV 1 RAPID TEST<br>TEST DATE: UNK | BIO-RAD MULTISPOT HIV 2 RAPID TEST<br>TEST DATE: UNK |
|--------------|--|---|--|--|--|--|
| 12007-01     | 0.07   | 0.276   | 0.09   | NEGATIVE   | NEGATIVE   | NEGATIVE   |
| 12007-02     | 0.09   | 0.276   | 0.13   | NEGATIVE   | NEGATIVE   | NEGATIVE   |
| 12007-03     | 0.06   | 0.256   | 0.12   | NEGATIVE   | NEGATIVE   | NEGATIVE   |
| 12007-04     | 2.33   | 1.281   | 3.94   | NEGATIVE   | NEGATIVE   | NEGATIVE   |
| 12007-05     | 2.10   | 1.882   | 3.35   | NEGATIVE   | NEGATIVE   | NEGATIVE   |
| 12007-06     | 4.71   | 14.773  | 14.72  | POSITIVE   | NEGATIVE   | NEGATIVE   |
| 12007-07     | 3.53   | 12.808  | 13.50  | POSITIVE   | NEGATIVE   | NEGATIVE   |
| 12007-08     | 2.23   | 14.419  | 7.79   | POSITIVE   | POSITIVE   | NEGATIVE   |
| 12007-09     | 2.11   | 12.813  | 5.63   | POSITIVE   | POSITIVE   | NEGATIVE   |

| PANEL MEMBER | ABBOTT HIV Ab ½ EIA rDNA (OD/Co)<br>TEST DATE: UNK | ABBOTT PRISM ANTI-HIV ½ +O<br>TEST DATE: 7/15/2002 | BIOMERIEUX VIRONOSTIKA ANTI-HIV-1<br>TEST DATE: UNK | BIO-RAD ANTI-HIV-1 rLAV<br>TEST DATE: UNK | BIOTEST ANTI-HIV TETRA ELISA<br>TEST DATE: 7/1/2002 | DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS<br>TEST DATE: 7/1/2002 | ORTHO ANTI-HIV ½ ENHANCED ELISA<br>TEST DATE: 6/24/2002 |
|--------------|--|--|---|---|---|---|---|
| 12007-01     | 0.095  | 0.28   | 0.485   | 0.069                                     | 0.12  | 0.03  | 0.00  |
| 12007-02     | 0.129  | 0.30   | 0.453   | 0.055                                     | 0.14  | 0.04  | 0.00  |
| 12007-03     | 0.121  | 0.27   | 0.515   | 0.044                                     | 0.15  | 0.05  | 0.00  |
| 12007-04     | 0.129  | 0.33   | 0.390   | 0.047                                     | 0.21  | 0.04  | 0.03  |
| 12007-05     | 1.147  | 0.60   | 0.347   | 0.051                                     | 0.89  | 0.39  | 1.23  |
| 12007-06     | 17.241   | 17.69  | 0.402   | 0.724                                     | 9.17  | OVER  | 24.40   |
| 12007-07     | 17.241   | 17.87  | 0.669   | 1.204                                     | 9.32  | OVER  | 24.14   |
| 12007-08     | 8.379  | 13.49  | 0.653   | 1.764                                     | 6.95  | 7.01  | 20.00   |
| 12007-09     | 6.957  | 18.07  | 0.656   | 2.309                                     | 6.30  | 7.18  | 18.10   |

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV12007 Rev06  
Effective Date: 10/26/2021