

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 65790 Part Number: HIV9016

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 8/20/1998	ABBOTT HIV-1 p24 Ag TEST DATE: 10/16/1998	ABBOTT HIV-1 p24 Ag NT% TEST DATE: 9/3/1998	COULTER HIV-1 p24 Ag TEST DATE: 10/3/1998	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: 9/16/1998	ABBOTT HIV AB ½ EIA rDNA TEST DATE: 8/27/1998	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 3/18/2015
9016-01	12/11/97	<50	0.301	NEGATIVE	0.05	0.34	0.256	0.293	0.16
9016-02	12/13/97	<50	0.260	NEGATIVE	0.79	0.35	0.171	0.272	0.16
9016-03	12/18/97	<50	0.260	NEGATIVE	0.05	0.35	0.256	0.283	0.16
9016-04	12/20/97	<50	0.288	NEGATIVE	0.05	0.37	0.171	0.283	0.16
9016-05	12/26/97	<50	0.247	NEGATIVE	0.07	0.34	0.171	0.283	0.16
9016-06	12/29/97	<50	0.247	NEGATIVE	0.08	0.34	0.171	0.297	0.16
9016-07	01/03/98	<50	0.260	NEGATIVE	0.05	0.36	0.171	0.318	0.16
9016-08	01/07/98	7473	0.288	NEGATIVE	0.32	0.35	0.171	0.339	0.16
9016-09	01/10/98	69010	0.767	86.60	3.36	0.33	0.256	0.297	0.16
9016-10	01/14/98	286400	3.014	99.00	13.22	0.38	0.171	0.300	0.27

PANEL MEMBER	ORTHO ECi ANTI-HIV ½ TEST DATE: UNK	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 8/18/1998	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: UNK	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 TEST DATE: 8/26/1998	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 9/2/1998	BIOTEST ANTI-HIV ½ TEST DATE: 4/22/1998	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 8/31/1998
9016-01	0.06	0.25	<3.0	NEGATIVE	0.50	0.39	0.010	0.04
9016-02	0.04	0.22	<3.0	NEGATIVE	0.53	0.40	0.000	0.02
9016-03	0.06	0.23	<3.0	NEGATIVE	0.48	0.40	0.015	0.03
9016-04	0.06	0.21	<3.0	NEGATIVE	0.47	0.39	0.003	0.03
9016-05	0.06	0.21	<3.0	NEGATIVE	0.47	0.42	0.007	0.02
9016-06	0.06	0.22	<3.0	NEGATIVE	0.52	0.42	0.002	0.02
9016-07	0.06	0.22	<3.0	NEGATIVE	0.50	0.44	0.021	0.02
9016-08	0.06	0.22	<3.0	NEGATIVE	0.50	0.41	0.007	0.14
9016-09	0.06	0.22	<3.0	NEGATIVE	0.52	0.43	0.000	0.03
9016-10	0.06	0.28	19.2	NEGATIVE	0.50	0.40	0.002	0.02

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9016 Rev06
Effective Date: 06/15/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 65790 Part Number: HIV9016

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	GENETICS SYSTEMS HIV ½ EIA TEST DATE: 8/25/1998	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9016-01	0.102	0.02	0.06	0.330	0.13	0.08	0.08	0.06
9016-02	0.080	0.01	0.06	0.249	0.07	0.07	0.07	0.02
9016-03	0.083	0.00	0.10	0.249	0.11	0.07	0.07	0.02
9016-04	0.083	0.01	0.07	0.279	0.14	0.10	0.10	0.02
9016-05	0.077	0.01	0.07	0.264	0.11	0.12	0.12	0.06
9016-06	0.089	0.02	0.06	0.279	0.15	0.09	0.09	0.06
9016-07	0.096	0.02	0.09	0.315	0.10	0.08	0.08	0.03
9016-08	0.086	0.01	0.20	0.381	0.21	0.67	0.67	0.05
9016-09	0.083	0.02	1.11	1.345	1.41	5.54	5.54	0.04
9016-10	0.099	0.02	5.94	6.340	4.45	34.36	34.36	0.05

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9016 Rev06
Effective Date: 06/15/2021