

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 66575 Part Number: HIV9018

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation	ORTHO VITROS ECi ANTI-HIV 1+2
		TEST DATE: 8/20/1998	TEST DATE: 8/27/1998	TEST DATE: 9/3/1998	TEST DATE: 10/3/1998	TEST DATE: 9/16/1998	TEST DATE: 6/29/1998	TEST DATE: UNK	TEST DATE: 3/23/2015
9018-01	02/11/98	<50	0.81	NEGATIVE	0.09	0.34	0.684	0.293	0.19
9018-02	02/15/98	<50	0.27	NEGATIVE	0.08	0.35	0.256	0.325	0.16
9018-03	02/18/98	<50	0.27	NEGATIVE	0.11	0.35	0.171	0.357	0.16
9018-04	02/22/98	<50	0.27	NEGATIVE	0.08	0.35	0.256	0.314	0.17
9018-05	02/25/98	<50	0.27	NEGATIVE	0.17	0.34	0.256	0.300	0.16
9018-06	03/01/98	<50	0.27	NEGATIVE	0.33	0.33	0.256	0.293	0.17
9018-07	03/04/98	304	0.27	NEGATIVE	0.09	0.33	0.256	0.283	0.16
9018-08	03/08/98	15280	0.54	NEGATIVE	0.21	0.35	0.256	0.286	0.16
9018-09	03/11/98	193100	3.38	97.60	1.55	0.35	0.427	0.396	0.19
9018-10	03/15/98	621100	8.51	98.40	3.43	2.59	11.880	9.201	10.8
9018-11	03/18/98	>500000	9.19	97.30	4.11	7.32	>17.097	12.580	24.8

PANEL MEMBER	ABBOTT PRISM ANTI-HIV ½	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0	BIOTEST ANTI-HIV ½	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS	GENETICS SYSTEMS HIV ½ EIA
	TEST DATE: 8/18/1998	TEST DATE: 8/26/1998	TEST DATE: 9/2/1998	TEST DATE: 9/24/1998	TEST DATE: 8/31/1998	TEST DATE: 8/25/1998
9018-01	0.24	0.48	0.45	0.044	0.03	0.121
9018-02	0.25	0.48	0.41	0.046	0.03	0.093
9018-03	0.26	0.55	0.42	0.052	0.02	0.089
9018-04	0.24	0.47	0.42	0.046	0.02	0.086
9018-05	0.25	0.52	0.44	0.032	0.02	0.086
9018-06	0.25	0.50	0.43	0.021	0.02	0.118
9018-07	0.24	0.53	0.51	0.031	0.03	0.099
9018-08	0.25	0.52	0.49	0.040	0.04	0.099
9018-09	0.29	0.50	0.48	0.052	0.04	0.105
9018-10	2.33	0.50	1.59	0.347	5.68	0.163
9018-11	17.57	0.48	4.18	2.408	MAX ABS.	0.693

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9018 Rev08
Effective Date: 10/25/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 66575 Part Number: HIV9018

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	WELLCOZYME ANTI-HIV ½ TEST DATE: 8/21/1998	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK
9018-01	0.02	0.358	NEGATIVE	0.06	0.259	0.13
9018-02	0.02	0.342	NEGATIVE	0.05	0.264	0.15
9018-03	0.01	0.346	NEGATIVE	0.06	0.254	0.12
9018-04	0.03	0.360	NEGATIVE	0.09	0.264	0.19
9018-05	0.03	0.357	NEGATIVE	0.09	0.254	0.14
9018-06	0.01	0.369	NEGATIVE	0.07	0.254	0.10
9018-07	0.03	0.350	NEGATIVE	0.07	0.244	0.14
9018-08	0.05	0.377	NEGATIVE	0.44	0.675	0.78
9018-09	0.07	0.376	NEGATIVE	4.18	5.015	5.46
9018-10	4.56	1.384	NEGATIVE	10.54	17.579	12.06
9018-11	9.62	4.283	INDETERMINATE	12.50	18.137	20.02

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9018 Rev08
Effective Date: 10/25/2021