

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 65685 Part Number: HIV9019

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 8/20/1998	ABBOTT HIV-1 p24 Ag TEST DATE: 7/21/1998	ABBOTT HIV-1 p24 Ag NT% TEST DATE: 9/3/1998	COULTER HIV-1 p24 Ag TEST DATE: 8/26/1998	ABBOTT AxsYM HIV AB ½ EIA TEST DATE: 9/16/1998	ABBOTT HIV AB ½ EIA rDNA TEST DATE: 7/21/1998	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation TEST DATE: UNK
9019-01	11/20/97	<50	0.317	NEGATIVE	0.64	0.35	0.256	0.314
9019-02	11/23/97	<50	0.317	NEGATIVE	0.56	0.39	0.171	0.258
9019-03	12/28/97	571700	1.571	NEGATIVE	1.44	7.54	>17.094	12.728

PANEL MEMBER	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 3/23/2015	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 8/18/1998	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 TEST DATE: UNK	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 9/2/1998	BIOTEST ANTI-HIV ½ TEST DATE: 9/24/1998	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 8/31/1998	GENETICS SYSTEMS HIV ½ EIA TEST DATE: 8/25/1998
9019-01	0.15	0.23	0.50	0.43	0.052	0.02	0.150
9019-02	0.16	0.26	0.50	0.83	0.018	0.01	0.150
9019-03	21.0	70.38	2.33	7.67	6.797	5.37	8.147

PANEL MEMBER	GENETICS SYSTEMS HIV ½ + O TEST DATE: UNK	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	ORAQUICK ADVANCE HIV ½ RAPID Ab TEST DATE: UNK	TRINITY BIOTECH UNIGOLD RECOMBIGN HIV-1 TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK
9019-01	0.168	0.05	P24 (VF)	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.17
9019-02	0.136	0.05	P24 (VF)	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.06
9019-03	9.709	11.0	P24 P55 P66 P120	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	10.54

PANEL MEMBER	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIORAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9019-01	0.259	0.08	0.16	0.16	0.05
9019-02	0.234	0.07	0.07	0.5	0.04
9019-03	17.934	57.28	129.83	NOT REACTIVE-HIGH AB	129.83

Notes on Data:

- This panel is for **Research Use Only. Not for use in diagnostic procedures.**
- Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
- All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
- No preservatives have been added.
- All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9019 Rev05
Effective Date: 10/25/2021