



**HIV Seroconversion Panel Donor No. 67512
Part Number: HIV9020**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24Ag	ZEPTOMETRIX HIV p24 Ag (pg/mL)	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA
		TEST DATE: 10/26/1999	TEST DATE: 6/16/1999	TEST DATE: 8/5/1999	TEST DATE: 6/21/1999	TEST DATE: UNK	TEST DATE: 7/14/1999	TEST DATE: 5/4/1999
9020-01	03/18/98	<50	0.44	0.490	0.52	<3.0	0.22	0.108
9020-02	03/23/98	<50	0.41	0.490	0.30	<3.0	0.19	0.090
9020-03	03/25/98	<50	0.44	0.490	0.32	<3.0	0.19	0.099
9020-04	03/30/98	<50	0.40	0.588	0.31	<3.0	0.20	0.090
9020-05	04/10/98	<50	0.38	0.490	0.32	<3.0	0.19	0.081
9020-06	04/12/98	<50	0.39	0.588	0.34	<3.0	0.24	0.081
9020-07	05/01/98	<50	0.35	0.490	0.32	<3.0	0.21	0.099
9020-08	05/03/98	<50	0.43	0.490	0.28	<3.0	0.20	0.081
9020-09	05/09/98	<50	0.46	0.490	0.36	<3.0	0.19	0.090
9020-10	05/11/98	<50	0.49	0.490	0.40	<3.0	0.24	0.090
9020-11	05/16/98	<50	0.13	0.490	0.32	<3.0	0.21	0.090
9020-12	05/18/98	<50	0.41	0.490	0.37	<3.0	0.21	0.081
9020-13	05/23/98	<50	0.48	0.588	0.34	<3.0	0.22	0.090
9020-14	05/25/98	<50	0.52	0.588	0.40	<3.0	0.24	0.090
9020-15	05/30/98	<50	0.38	0.490	0.42	<3.0	0.22	0.108
9020-16	06/01/98	<50	0.36	0.490	0.40	<3.0	0.23	0.099
9020-17	06/06/98	<50	0.41	0.588	0.34	<3.0	0.20	0.081
9020-18	06/09/98	59	0.44	0.588	0.40	<3.0	0.20	0.072
9020-19	06/13/98	8548	0.54	1.020	0.65	<3.0	0.19	0.099
9020-20	06/16/98	69238	1.01	5.098	3.04	<3.0	0.23	0.099
9020-21	06/20/98	145718	1.63	9.510	4.90	6.3	0.39	0.351
9020-22	06/23/98	178847	1.40	10.294	4.31	4.2	0.79	1.595

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9020 Rev05
Effective Date: 10/25/2021



**HIV Seroconversion Panel Donor No. 67512
Part Number: HIV9020**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA 3 rd Generation TEST DATE: 6/22/1999	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: 7/19/1999	ABBOTT MUREX HIV Ab ½ + O TEST DATE: UNK	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 3/23/2015	ABBOTT MUREX SUDS ANTI-HIV-1 TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM TEST DATE: 6/16/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 6/9/1999
9020-01	0.24	0.26	0.261	0.18	0.15	0.20	0.346	0.39
9020-02	0.20	0.28	0.269	0.18	0.18	0.26	0.317	0.34
9020-03	0.23	0.28	0.290	0.18	0.16	0.24	0.317	0.36
9020-04	0.19	0.28	0.343	0.18	0.15	0.27	0.346	0.37
9020-05	0.23	0.27	0.336	0.18	0.15	0.27	0.317	0.00
9020-06	0.23	0.30	0.293	0.19	0.19	0.22	0.317	0.53
9020-07	0.23	0.33	0.300	0.17	0.19	0.26	0.951	0.39
9020-08	0.23	0.28	0.258	0.21	0.16	0.23	0.634	0.40
9020-09	0.22	0.31	0.272	0.18	0.17	0.25	0.432	0.00
9020-10	0.26	0.35	0.258	0.17	0.22	0.26	0.403	0.47
9020-11	0.24	0.38	0.279	0.17	0.24	0.25	0.346	0.41
9020-12	0.26	0.34	0.300	0.17	0.16	0.35	0.317	0.36
9020-13	0.24	0.33	0.353	0.15	0.16	0.27	0.317	0.41
9020-14	0.20	0.34	0.293	0.16	0.19	0.24	0.346	0.41
9020-15	0.23	0.32	0.297	0.16	0.17	0.24	0.317	0.40
9020-16	0.27	0.34	0.279	0.17	0.17	0.29	0.288	0.40
9020-17	0.27	0.33	0.276	0.19	0.21	0.25	0.259	0.38
9020-18	0.20	0.31	0.258	0.18	0.17	0.25	0.346	0.40
9020-19	0.26	0.32	0.279	0.16	0.16	0.27	0.375	0.36
9020-20	0.29	0.29	0.322	0.17	0.16	0.26	0.403	0.38
9020-21	0.56	0.31	0.915	0.25	0.83	0.48	1.239	0.44
9020-22	1.63	1.41	3.597	0.22	2.42	2.78	1.816	0.87

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9020 Rev05
Effective Date: 10/25/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 67512 Part Number: HIV9020

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT	BIOTEST ANTI-HIV ½	BIOTEST ANTI-HIV-1 TETRA ELISA	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS	GENETICS SYSTEMS HIV ½ PEPTIDE EIA	GENETICS SYSTEMS rLAV EIA	ORTHO ANTI-HIV ½ ENHANCED ELISA	ROCHE ENZYMUM ANTI-HIV- COMBI
	TEST DATE: UNK	TEST DATE: 6/9/1999	TEST DATE: 6/8/1999	TEST DATE: 6/14/1999	TEST DATE: 6/10/1999	TEST DATE: 11/15/1999	TEST DATE: 6/21/1999	TEST DATE: 7/14/1999
9020-01	NEGATIVE	0.03	0.20	0.10	0.286	0.112	0.03	0.10
9020-02	NEGATIVE	0.03	0.24	0.15	0.314	0.192	0.05	0.08
9020-03	NEGATIVE	0.03	0.23	0.13	0.257	0.109	0.04	0.10
9020-04	NEGATIVE	0.06	0.21	0.13	0.200	0.106	0.04	0.10
9020-05	NEGATIVE	0.03	0.23	0.13	0.143	0.145	0.08	0.09
9020-06	NEGATIVE	0.05	0.22	0.13	0.257	0.145	0.05	0.08
9020-07	NEGATIVE	0.03	0.25	0.12	0.314	0.130	0.05	0.08
9020-08	NEGATIVE	0.03	0.25	0.13	0.457	0.124	0.05	0.09
9020-09	NEGATIVE	0.03	0.21	0.12	0.257	0.109	0.07	0.10
9020-10	NEGATIVE	0.03	0.24	0.12	0.229	0.086	0.05	0.10
9020-11	NEGATIVE	0.03	0.19	0.12	0.229	0.103	0.05	0.09
9020-12	NEGATIVE	0.02	0.21	0.14	0.200	0.074	0.07	0.10
9020-13	NEGATIVE	0.02	0.22	0.14	0.286	0.115	0.05	0.09
9020-14	NEGATIVE	0.02	0.23	0.14	0.286	0.121	0.06	0.08
9020-15	NEGATIVE	0.03	0.21	0.12	0.457	0.136	0.05	0.09
9020-16	NEGATIVE	0.04	0.22	0.12	0.229	0.118	0.02	0.10
9020-17	NEGATIVE	0.03	0.21	0.12	0.229	0.112	0.05	0.01
9020-18	NEGATIVE	0.04	0.21	0.12	0.143	0.118	0.05	0.09
9020-19	NEGATIVE	0.03	0.19	0.12	0.229	0.121	0.01	0.08
9020-20	NEGATIVE	0.04	0.21	0.13	0.257	0.133	0.04	0.11
9020-21	NEGATIVE	0.80	0.58	0.64	0.257	0.071	0.15	2.27
9020-22	NEGATIVE	2.30	2.75	1.95	0.314	0.100	1.65	27.40

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9020 Rev05
Effective Date: 10/25/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 67512 Part Number: HIV9020

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOFI GENSCREEN ANTI-HIV ½ TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9020-01	0.06	0.10	0.264	0.09	0.08	0.08	0.00
9020-02	0.05	0.10	0.239	0.09	0.04	0.04	0.00
9020-03	0.04	0.07	0.244	0.09	0.08	0.08	0.00
9020-04	0.07	0.20	0.305	0.11	0.07	0.05	0.04
9020-05	0.07	0.18	0.249	0.09	0.03	0.03	0.01
9020-06	0.07	0.09	0.254	0.09	0.05	0.05	0.03
9020-07	0.04	0.06	0.249	0.08	0.06	0.06	0.01
9020-08	0.04	0.07	0.234	0.09	0.05	0.04	0.03
9020-09	0.04	0.06	0.239	0.10	0.13	0.13	0.04
9020-10	0.04	0.08	0.228	0.10	0.13	0.13	0.04
9020-11	0.05	0.06	0.244	0.11	0.06	0.06	0.01
9020-12	0.06	0.05	0.264	0.09	0.07	0.07	0.06
9020-13	0.06	0.07	0.249	0.08	0.07	0.07	0.06
9020-14	0.05	0.05	0.249	0.10	0.12	0.12	0.06
9020-15	0.04	0.05	0.264	0.08	0.09	0.09	0.03
9020-16	0.06	0.09	0.239	0.07	0.03	0.02	0.03
9020-17	0.06	0.06	0.239	0.08	0.06	0.03	0.05
9020-18	0.04	0.06	0.264	0.10	0.09	0.09	0.03
9020-19	0.04	0.32	0.376	0.35	0.50	0.50	0.01
9020-20	0.10	1.70	1.797	3.05	5.93	5.93	0.06
9020-21	13.57	4.09	3.574	4.34	9.88	9.88	0.10
9020-22	14.69	3.10	4.624	6.49	8.66	8.66	3.36

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9020 Rev05
Effective Date: 10/25/2021