



**HIV Seroconversion Panel Donor No. 67706  
Part Number: HIV9023**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation	ABBOTT MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation
		TEST DATE: 10/26/1999	TEST DATE: 6/16/1999	TEST DATE: 6/18/1999	TEST DATE: 6/21/1999	TEST DATE: 7/14/1999	TEST DATE: 5/13/1999	TEST DATE: 7/12/1999	TEST DATE: UNK
9023-01	04/15/98	<50	0.37	0.602	0.28	0.28	0.157	0.26	0.366
9023-02	04/17/98	<50	0.39	0.482	0.33	0.20	0.182	0.31	0.325
9023-03	04/22/98	<50	0.38	0.482	0.23	0.22	0.165	0.27	0.328
9023-04	04/24/98	<50	0.41	0.482	0.26	0.21	0.149	0.26	0.375
9023-05	04/29/98	<50	0.41	0.482	0.24	0.20	0.190	0.29	0.378
9023-06	05/01/98	<50	0.45	0.361	0.91	0.20	0.182	0.28	0.472
9023-07	05/13/98	<50	0.38	0.361	0.24	0.21	0.207	0.31	0.388
9023-08	05/15/98	<50	0.46	0.361	0.28	0.21	0.182	0.28	0.372
9023-09	05/20/98	<50	0.29	0.241	0.26	0.23	0.165	0.27	0.325
9023-10	05/22/98	<50	0.44	0.482	0.24	0.22	0.174	0.32	0.309
9023-11	05/27/98	<50	0.40	0.361	0.28	0.22	0.190	0.33	0.316
9023-12	05/29/98	<50	0.43	0.482	0.29	0.21	0.182	0.24	0.359
9023-13	06/09/98	<50	0.40	0.602	0.24	0.20	0.174	0.27	0.428
9023-14	06/11/98	<50	0.50	0.361	0.31	0.22	0.182	0.27	0.522
9023-15	06/16/98	<50	0.45	0.361	0.29	0.21	0.165	0.28	0.403
9023-16	06/18/98	<50	0.39	0.361	0.36	0.22	0.165	0.26	0.356
9023-17	06/23/98	<50	0.40	0.361	0.33	0.20	0.174	0.28	0.331
9023-18	06/25/98	<50	0.46	0.361	0.33	0.20	0.182	0.31	0.319
9023-19	06/30/98	7183	0.45	0.602	0.50	0.22	0.149	0.26	0.309
9023-20	07/02/98	62566	0.73	2.048	2.16	0.21	0.149	0.28	0.381
9023-21	07/07/98	>500000	7.98	38.554	56.14	0.20	0.182	0.24	0.400
9023-22	07/09/98	>500000	11.88	40.964	83.63	0.21	0.198	0.34	0.488

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 67706  
Part Number: HIV9023**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA IMx  TEST DATE: 7/19/1999	ABBOTT MUREX ANTI-HIV ½ ICE + O  TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV ½  TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM  TEST DATE: 6/18/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0  TEST DATE: 6/9/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT  TEST DATE: UNK	BIOTEST ANTI-HIV ½  TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA  TEST DATE: 6/8/1999
9023-01	0.47	0.20	0.31	0.267	0.46	NEGATIVE	0.06	0.27
9023-02	0.38	0.17	0.23	0.267	0.58	NEGATIVE	0.05	0.25
9023-03	0.31	0.17	0.22	0.297	0.51	NEGATIVE	0.05	0.27
9023-04	0.35	0.17	0.24	0.267	0.48	NEGATIVE	0.03	0.23
9023-05	0.39	0.16	0.24	0.267	0.45	NEGATIVE	0.05	0.25
9023-06	0.46	0.17	0.22	0.267	0.42	NEGATIVE	0.04	0.23
9023-07	0.36	0.15	0.22	0.237	0.44	NEGATIVE	0.05	0.33
9023-08	0.39	0.16	0.25	0.237	0.44	NEGATIVE	0.05	0.24
9023-09	0.36	0.29	0.19	0.237	0.46	NEGATIVE	0.04	0.28
9023-10	0.33	0.17	0.31	0.267	0.45	NEGATIVE	0.05	0.30
9023-11	0.42	0.15	0.22	0.267	0.51	NEGATIVE	0.08	0.24
9023-12	0.38	0.16	0.45	0.267	0.46	NEGATIVE	0.05	0.25
9023-13	0.41	0.17	0.24	0.267	0.48	NEGATIVE	0.07	0.26
9023-14	0.34	0.15	0.25	0.267	0.50	NEGATIVE	0.06	0.25
9023-15	0.34	0.15	0.24	0.297	0.47	NEGATIVE	0.06	0.26
9023-16	0.33	0.16	0.31	0.267	0.51	NEGATIVE	0.05	0.31
9023-17	0.37	0.19	0.25	0.267	0.49	NEGATIVE	0.05	0.25
9023-18	0.36	0.18	0.23	0.237	0.51	NEGATIVE	0.05	0.29
9023-19	0.31	0.16	0.23	0.237	0.51	NEGATIVE	0.07	0.27
9023-20	0.34	0.14	0.24	0.237	0.50	NEGATIVE	0.07	0.30
9023-21	0.32	0.14	0.24	0.237	0.41	NEGATIVE	0.06	0.24
9023-22	0.34	0.24	0.26	0.237	0.45	NEGATIVE	0.06	0.55

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 67706  
Part Number: HIV9023**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA TEST DATE: 6/28/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/29/1999	ROCHE ENZYMUM ANTI-HIV-COMBI TEST DATE: 7/14/1999	SANOFI GENSCREEN ANTI-HIV ½ TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK
9023-01	0.13	0.211	0.056	0.05	0.09	0.06	0.11	0.213
9023-02	0.12	0.211	0.074	0.05	0.08	0.05	0.10	0.213
9023-03	0.10	0.211	0.065	0.05	0.09	0.05	0.11	0.223
9023-04	0.03	0.141	0.065	0.05	0.08	0.05	0.07	0.228
9023-05	0.10	0.317	0.053	0.03	0.09	0.08	0.08	0.279
9023-06	0.12	0.352	0.059	0.04	0.08	0.07	0.09	0.279
9023-07	0.13	0.458	0.062	0.04	0.09	0.06	0.05	0.467
9023-08	0.15	0.176	0.056	0.05	0.09	0.04	0.08	0.274
9023-09	0.15	0.106	0.053	0.05	0.08	0.05	0.06	0.218
9023-10	0.12	0.141	0.065	0.05	0.09	0.06	0.06	0.223
9023-11	0.11	0.106	0.062	0.04	0.08	0.04	0.06	0.213
9023-12	0.11	0.176	0.074	0.04	0.08	0.04	0.11	0.218
9023-13	0.11	0.070	0.068	0.05	0.08	0.06	0.06	0.239
9023-14	0.11	0.106	0.071	0.05	0.08	0.07	0.07	0.254
9023-15	0.14	0.211	0.077	0.06	0.10	0.06	0.09	0.340
9023-16	0.12	0.141	0.062	0.05	0.10	0.05	0.09	0.218
9023-17	0.13	0.106	0.062	0.04	0.10	0.07	0.08	0.223
9023-18	0.13	0.106	0.065	0.06	0.09	0.05	0.07	0.223
9023-19	0.04	0.141	0.065	0.06	0.10	0.07	0.23	0.305
9023-20	0.12	0.106	0.074	0.04	0.11	0.03	0.82	1.056
9023-21	0.12	0.176	0.059	0.09	1.83	0.13	26.41	17.523
9023-22	0.13	0.211	0.056	1.18	2.91	6.75	37.38	18.914

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 67706  
Part Number: HIV9023**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/30/2018
9023-01	0.11	0.09	0.09	0.04	0.10
9023-02	0.11	0.08	0.08	0.04	0.09
9023-03	0.13	0.10	0.10	0.04	0.09
9023-04	0.12	0.07	0.07	0.01	0.10
9023-05	0.14	0.04	0.03	0.01	0.10
9023-06	0.12	0.15	0.15	0.05	0.10
9023-07	0.13	0.04	0.04	0.01	0.11
9023-08	0.15	0.07	0.06	0.04	0.10
9023-09	0.13	0.11	0.11	0.04	0.10
9023-10	0.13	0.06	0.06	0.04	0.10
9023-11	0.13	0.08	0.08	0.02	0.10
9023-12	0.12	0.08	0.08	0.06	0.10
9023-13	0.13	0.05	0.05	0.01	0.10
9023-14	0.35	0.15	0.15	0.05	0.11
9023-15	0.12	0.08	0.08	0.03	0.10
9023-16	0.12	0.11	0.11	0.07	0.10
9023-17	0.11	0.03	0.02	0.02	0.10
9023-18	0.13	0.11	0.11	0.09	0.10
9023-19	0.30	0.21	0.21	0.05	0.09
9023-20	1.94	3.73	3.73	0.02	0.10
9023-21	58.51	98.83	98.83	0.05	0.12
9023-22	87.79	125.10	125.10	0.08	0.84

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)