



**HIV Seroconversion Panel Donor No. 66686**  
**Part Number: HIV9033**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	ROCHE ELECSYS HIV p24 Ag	COULTER HIV-1 p24 Ag	ABBOTT HIV Ag 1 MONOCLONAL HIV AB ½ EIA	ABBOTT HIV Ab ½ EIA rDNA	ABBOTT HIV Ab ½ EIA rDNA 3 <sup>RD</sup> Generation
		TEST DATE: 10/26/1999	TEST DATE: 5/28/1999	TEST DATE: 8/16/2000	TEST DATE: 6/10/1999	TEST DATE: 8/29/2000	TEST DATE: 5/28/1999	TEST DATE: 11/15/1999
9033-01	01/04/98	<50	0.173	0.24	0.190	0.29	0.153	0.24
9033-02	01/15/98	<50	0.173	0.25	0.165	0.33	0.127	0.24
9033-03	01/18/98	<50	0.212	0.27	0.177	0.33	0.136	0.23
9033-04	01/22/98	<50	0.173	0.26	0.253	0.31	0.144	0.23
9033-05	01/25/98	<50	0.192	NEGATIVE	0.253	0.31	0.144	0.25
9033-06	01/29/98	<50	0.212	0.24	0.228	0.34	0.153	0.24
9033-07	02/03/98	<50	0.173	0.28	NEGATIVE	0.30	0.144	0.22
9033-08	02/06/98	<50	0.192	0.23	0.266	0.28	0.127	0.24
9033-09	02/13/98	<50	0.154	0.27	0.291	0.31	0.161	0.23
9033-10	02/16/98	<50	0.096	0.25	0.241	0.33	0.153	0.23
9033-11	02/22/98	<50	0.135	NEGATIVE	0.241	0.31	0.169	0.23
9033-12	02/26/98	<50	0.192	0.25	0.228	0.33	0.119	0.24
9033-13	03/05/98	<50	0.154	0.22	0.203	0.31	0.161	0.24
9033-14	03/08/98	<50	0.154	NEGATIVE	0.215	0.33	0.186	0.24
9033-15	03/27/98	128365	0.981	5.31	3.684	1.90	0.178	0.34
9033-16	03/29/98	454553	2.981	12.06	5.823	3.80	0.390	0.68

PANEL MEMBER	ABBOTT IMx HIV Ab ½ EIA rDNA	ABBOTT MUREX HIV Ab ½ + O 3 <sup>RD</sup> Generation	ABBOTT AxSYM ANTI-HIV ½ EIA	ABBOTT MUREX HIV ½ + O	BIOMERIEUX VIDAS HIV DUO	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0
	TEST DATE: 6/7/2000	TEST DATE: UNK	TEST DATE: 11/17/1999	TEST DATE: 8/21/2000	TEST DATE: 11/2/1999	TEST DATE: 4/19/2000
9033-01	0.46	0.340	0.29	0.27	0.20	0.41
9033-02	0.40	0.319	0.38	0.29	0.20	0.39
9033-03	0.35	0.288	0.24	0.35	0.16	0.38
9033-04	0.37	0.278	0.22	0.29	0.16	0.41
9033-05	0.39	0.285	0.22	0.32	0.20	0.39
9033-06	0.35	0.285	0.31	0.35	0.16	0.48
9033-07	0.42	0.288	0.39	0.33	0.16	0.50
9033-08	0.36	0.347	0.43	0.40	0.20	0.46
9033-09	0.36	0.361	0.23	0.35	0.16	0.41
9033-10	0.40	0.299	0.45	0.27	0.16	0.42
9033-11	0.37	0.299	0.65	0.36	0.20	0.42
9033-12	0.39	0.302	0.23	0.28	0.20	0.40
9033-13	0.48	0.271	0.27	0.38	0.20	0.42
9033-14	0.40	0.278	0.23	0.32	0.20	0.52
9033-15	0.80	0.569	0.37	0.46	1.04	0.56
9033-16	1.18	2.313	0.33	1.15	2.20	1.21

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 66686**  
**Part Number: HIV9033**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOR-RAD NOVAPATH HIV-1 IMMUNOBLOT TEST DATE: UNK	DADE BEHRING ENZYGNOST ANTI-HIV 1/2 PLUS TEST DATE: 11/18/1999	GENETICS SYSTEMS HIV 1/2 EIA TEST DATE: 6/10/1999	ORTHO VITROS ECI ANTI HIV 1/2 Ab CAPTURE ELISA TEST DATE: 4/4/2000	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK
9033-01	NEGATIVE	0.07	0.314	0.07	0.10	0.242
9033-02	NEGATIVE	0.06	0.314	0.07	0.08	0.237
9033-03	NEGATIVE	0.06	0.371	0.07	0.07	0.227
9033-04	NEGATIVE	0.07	0.229	0.07	0.09	0.227
9033-05	NEGATIVE	0.05	0.371	0.07	0.07	0.217
9033-06	NEGATIVE	0.05	0.200	0.07	0.08	0.212
9033-07	NEGATIVE	0.05	0.371	0.42	0.09	0.212
9033-08	NEGATIVE	0.01	0.343	0.1	0.12	0.212
9033-09	NEGATIVE	0.06	0.286	0.07	0.08	0.232
9033-10	NEGATIVE	0.06	0.143	0.07	0.11	0.222
9033-11	NEGATIVE	0.06	0.229	0.07	0.08	0.222
9033-12	NEGATIVE	0.06	0.457	0.07	0.08	0.222
9033-13	NEGATIVE	0.06	0.171	0.07	0.10	0.217
9033-14	NEGATIVE	0.06	0.229	0.07	0.10	0.217
9033-15	NEGATIVE	0.16	0.371	0.80	3.28	2.096
9033-16	NEGATIVE	0.48	0.314	3.82	5.77	5.399

PANEL MEMBER	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>TH</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation TEST DATE: UNK	ORTHO VITROS ECI ANTI-HIV 1/2 TEST DATE: 8/30/2018
9033-01	0.16	0.11	0.11	0.03	0.12
9033-02	0.07	0.07	0.07	0.05	0.12
9033-03	0.14	0.10	0.05	0.06	0.12
9033-04	0.10	0.06	0.05	0.05	0.11
9033-05	0.14	0.07	0.07	0.01	0.12
9033-06	0.15	0.11	0.11	0.04	0.12
9033-07	0.13	0.06	0.06	0.00	0.12
9033-08	0.08	0.11	0.11	0.00	0.16
9033-09	0.10	0.08	0.08	0.02	0.12
9033-10	0.10	0.05	0.05	0.02	0.12
9033-11	0.11	0.10	0.10	0.06	0.12
9033-12	0.06	0.06	0.06	0.01	0.12
9033-13	0.10	0.05	0.05	0.02	0.11
9033-14	0.11	0.07	0.07	0.03	0.14
9033-15	3.74	14.17	14.17	0.07	0.37
9033-16	5.95	33.64	33.64	0.04	2.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 66686**  
**Part Number: HIV9033**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

[www.h-h-c.com](http://www.h-h-c.com)

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)