



**HIV Seroconversion Panel Donor No. 75018  
Part Number: HIV9077**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN-PROBE PROCLEIX HIV-1/HCV ASSAY  TEST DATE: UNK	BIO-RAD HIV-1 WESTERN BLOT  TEST DATE: UNK	ZEPTOMETRIX HIV p24 Ag (pg/mL)  TEST DATE: UNK	ABBOTT HIV AB 1/2 EIA rDNA  TEST DATE: 6/3/2002	ABBOTT MUREX HIV Ab 1/2 + O 3 <sup>RD</sup> Generation  TEST DATE: UNK
9077-01	10/20/99	NEGATIVE	NOT TESTED	<3.0	0.153	0.511
9077-02	10/25/99	NEGATIVE	NOT TESTED	<3.0	0.144	0.461
9077-03	10/27/99	NEGATIVE	NOT TESTED	<3.0	0.162	0.393
9077-04	11/02/99	NEGATIVE	NOT TESTED	<3.0	0.162	0.396
9077-05	11/04/99	NEGATIVE	NOT TESTED	<3.0	0.144	0.368
9077-06	11/09/99	NEGATIVE	NOT TESTED	<3.0	0.144	0.427
9077-07	11/11/99	NEGATIVE	NOT TESTED	<3.0	0.135	0.412
9077-08	11/16/99	NEGATIVE	NOT TESTED	<3.0	0.126	0.511
9077-09	11/18/99	NEGATIVE	NOT TESTED	<3.0	0.135	0.545
9077-10	11/23/99	NEGATIVE	NOT TESTED	<3.0	0.162	0.458
9077-11	11/30/99	POSITIVE	NOT TESTED	<3.0	0.153	0.412
9077-12	12/04/99	POSITIVE	NEGATIVE	6.3	0.198	0.415
9077-13	12/09/99	POSITIVE	NEGATIVE	105.5	0.135	0.560
9077-14	12/11/99	POSITIVE	NEGATIVE	>125	0.432	1.003
9077-15	12/16/99	POSITIVE	GP160,P55,P40,P24	29.3	8.793	10.582
9077-16	12/18/99	POSITIVE	GP160,P55,P40,P24	4.8	6.721	10.573
9077-17	12/23/99	POSITIVE	GP160,P55,P40,P24	<3.0	9.514	10.477
9077-18	12/26/99	POSITIVE	GP160,P55,P40,P24	<3.0	10.225	10.551
9077-19	12/30/99	POSITIVE	GP160,P65+/-,P55,P40,P24,P18+/-	<3.0	11.919	10.362
9077-20	01/02/00	POSITIVE	GP160,P65,P55,P40,P31+/-,P24,P18+/-	<3.0	12.730	10.384
9077-21	01/06/00	POSITIVE	GP160,P65,P55,P40,P31,P24,P18+/-	<3.0	13.649	10.421
9077-22	01/09/00	POSITIVE	GP160,P65,P55,P40,P31,P24,P18	<3.0	13.757	10.511
9077-23	01/13/00	POSITIVE	GP160,P65,P55,P40,P31,P24,P18	<3.0	15.009	NOT TESTED
9077-24	01/16/00	POSITIVE	GP160,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	15.306	10.517
9077-25	01/20/00	POSITIVE	GP160,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	17.820	10.885
9077-26	01/23/00	POSITIVE	GP160,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.833
9077-27	01/27/00	POSITIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.721
9077-28	01/30/00	POSITIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.771
9077-29	02/03/00	POSITIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.780

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9077 Rev07  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 75018  
Part Number: HIV9077**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOMERIEUX VIRONOSTIKA ANTI HIV-1 TEST DATE: UNK	BIO-RAD ANTI HIV-1 rLAV TEST DATE: 6/3/2002	GENETICS SYSTEMS HIV 1/2 + O TEST DATE: UNK	TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1 TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	MEDMIRA ANTI-HIV-1 RAPID TEST TEST DATE: UNK	ORAQUICK ADVANCE HIV 1/2 RAPID Ab TEST DATE: UNK
9077-01	0.421	0.055	0.334	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-02	0.348	0.066	0.133	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-03	0.308	0.033	0.191	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-04	0.368	0.063	0.133	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-05	0.325	0.055	0.154	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-06	0.375	0.041	0.106	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-07	0.461	0.052	0.106	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-08	0.700	0.055	0.109	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-09	0.453	0.052	0.304	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-10	0.338	0.066	0.119	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-11	0.287	0.066	0.191	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-12	0.408	0.081	0.119	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-13	0.392	0.066	0.526	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-14	0.405	0.081	5.280	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
9077-15	1.598	0.771	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-16	2.833	1.373	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-17	2.107	1.642	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-18	2.437	1.716	10.239	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-19	1.927	1.875	10.239	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-20	2.081	2.148	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-21	1.912	2.336	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-22	2.014	2.708	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-23	2.477	2.993	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-24	4.335	3.421	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-25	2.844	3.779	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-26	2.793	3.996	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-27	3.273	3.343	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-28	3.225	4.033	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-29	3.606	5.413	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9077 Rev07  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 75018  
Part Number: HIV9077**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>TH</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV 1/2  TEST DATE: 8/4/2015
9077-01	0.17	0.240	0.09	0.10	0.10	0.01	0.13
9077-02	0.06	0.250	0.28	0.07	0.07	0.03	0.13
9077-03	0.10	0.240	0.09	0.05	0.03	0.04	0.14
9077-04	0.06	0.245	0.19	0.13	0.13	0.02	0.13
9077-05	0.08	0.240	0.13	0.06	0.04	0.06	0.14
9077-06	0.08	0.255	0.15	0.08	0.08	0.03	0.13
9077-07	0.07	0.270	0.10	0.05	0.04	0.03	0.12
9077-08	0.07	0.270	0.22	0.07	0.07	0.06	0.15
9077-09	0.11	0.245	0.12	0.11	0.11	0.03	0.13
9077-10	0.10	0.235	0.11	0.06	0.04	0.05	0.13
9077-11	0.08	0.294	0.15	0.17	0.17	0.03	0.13
9077-12	2.19	2.245	2.83	10.00	10.00	0.04	0.14
9077-13	29.22	46.279	36.86	147.15	147.15	0.07	0.59
9077-14	37.81	16.980	42.80	166.91	166.91	0.86	2.43
9077-15	14.75	13.819	20.42	24.73	24.49	24.73	25.3
9077-16	5.66	9.078	17.99	25.27	4.02	25.27	29.4
9077-17	2.92	11.520	11.14	19.58	0.62	19.58	25.8
9077-18	2.71	13.583	8.45	17.18	0.49	17.18	21.9
9077-19	2.39	15.966	5.99	11.93	0.29	11.93	22.4
9077-20	2.82	16.471	5.38	10.73	0.19	10.73	25.2
9077-21	3.10	16.966	5.07	9.14	0.12	9.14	26.9
9077-22	3.47	16.863	5.64	9.54	0.07	9.54	32.2
9077-23	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-24	5.09	16.877	9.54	15.32	0.12	15.32	31.8
9077-25	5.92	18.206	15.37	25.19	0.09	25.19	36.3
9077-26	6.47	18.652	15.81	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-27	7.24	17.980	23.08	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-28	8.15	17.760	33.61	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-29	9.47	17.873	51.72	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9077 Rev07  
Effective Date: 10/26/2021