



**HIV Seroconversion Panel Donor No. 79065
Part Number: HIV9078**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN-PROBE PROCLEIX HIV-1 /HCV ASSAY TEST DATE: UNK	ABBOTT HIV-1 p24 Ag TEST DATE: 5/16/2001	COULTER HIV-1 p24 Ag TEST DATE: 5/16/2001	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: UNK	ABBOTT HIV AB 1/2 EIA rDNA TEST DATE: UNK	ABBOTT MUREX HIV Ab 1/2 + O 3 RD Generation TEST DATE: UNK	BIOMERIEUX VIRONOSTIKA ANTI-HIV-1 TEST DATE: UNK	BIO-RAD ANTI-HIV-1 rLAV TEST DATE: UNK
9078-01	05/22/00	NEGATIVE	0.8	0.31	<3.0	0.054	0.387	0.401	0.062
9078-02	05/24/00	NEGATIVE	0.3	0.20	<3.0	0.045	0.396	0.508	0.058
9078-03	07/09/00	NEGATIVE	0.3	0.26	<3.0	0.171	0.486	0.525	0.069
9078-04	07/17/00	NEGATIVE	0.3	0.28	<3.0	0.099	0.542	0.367	0.047
9078-05	07/30/00	NEGATIVE	0.5	0.30	<3.0	0.162	0.489	0.556	0.051

PANEL MEMBER	GENETICS SYSTEMS HIV 1/2 + O TEST DATE: UNK	TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1 TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	MEDMIRA ANTI-HIV-1 RAPID TEST TEST DATE: UNK	ORAQUICK ADVANCE HIV 1/2 RAPID Ab TEST DATE: UNK	BIO-RAD HIV-1 WESTERN BLOT TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK
9078-01	0.159	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.07
9078-02	0.142	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.11
9078-03	0.214	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE	0.07
9078-04	0.217	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE	0.09
9078-05	0.146	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE	0.06

PANEL MEMBER	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/30/2018
9078-01	0.260	0.18	0.06	0.06	0.01	0.11
9078-02	0.250	0.10	0.07	0.07	0.00	0.11
9078-03	0.260	0.08	0.09	0.09	0.04	0.11
9078-04	0.279	0.08	0.08	0.08	0.00	0.10
9078-05	0.265	0.10	0.09	0.09	0.03	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com