



Zika Virus Seroconversion Panel Donor No. PMR1001029
Product Code: KZMC047

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and symptomatic throughout the collection period.

PANEL MEMBER	BLEED DATE	EUROIMMUN Zika IgM (S/Co)	EUROIMMUN Zika IgG (Result)	EUROIMMUN ChikV IgM (S/Co)	EUROIMMUN ChikV IgG (Result)	EUROIMMUN Dengue IgM (S/Co)	EUROIMMUN Dengue IgG (Result)
KZMC047-01	8/23/2016	3.00	19.9 (IND)	0.18	1.8	0.39	8.0
KZMC047-02	8/26/2016	2.68	23.0	0.15	1.6	0.33	6.8
KZMC047-03	8/30/2016	2.02	22.0 (IND)	0.14	1.7	0.29	7.2
KZMC047-04	9/2/2016	1.74	35.6	0.10	1.6	0.23	8.5
KZMC047-05	9/6/2016	1.10 (IND)	26.3	0.09	1.6	0.17	6.1
KZMC047-06	9/10/2016	0.76	22.0 (IND)	0.07	1.4	0.15	6.1
KZMC047-07	9/13/2016	0.66	29.9	0.08	1.6	0.15	7.4

PANEL MEMBER	BLEED DATE	EUROIMMUN WNV IgM (S/Co)	EUROIMMUN WNV IgG (Result)	RT-PCR	DIACHECK Anti-Zika IgG S/Co	DIACHECK Anti-Zika IgM S/Co
KZMC047-01	8/23/2016	0.34	1.5	Undetectable	3.832	2.963
KZMC047-02	8/26/2016	0.36	1.4	Undetectable	4.106	2.923
KZMC047-03	8/30/2016	0.34	1.6	Undetectable	4.644	2.286
KZMC047-04	9/2/2016	0.26	1.6	Undetectable	4.261	1.769
KZMC047-05	9/6/2016	0.25	1.9	Undetectable	4.598	1.382
KZMC047-06	9/10/2016	0.24	2.3	Undetectable	4.684	1.012
KZMC047-07	9/13/2016	0.26	2.5	Undetectable	4.024	0.609

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures..**
2. Data is generated at ZeptoMetrix™ LLC and by our Lab Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No Preservatives have been added.

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Use Universal Precautions!