



HBV Seroconversion Panel Donor No. 62433

Part Number: HBV6281

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3	ROCHE Cobas Core HBsAg II EIA
		TEST DATE: 1/27/1998	TEST DATE: UNK	TEST DATE: 6/8/1998	TEST DATE: 4/22/1998	TEST DATE: 4/23/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 4/23/1998
6281-01	09-03-96	4	92	100	0.12	0.40	0.41	0.55	0.13
6281-02	09-08-96	4	133	200	0.14	0.54	0.55	0.41	0.24
6281-03	09-10-96	0	593	500	0.11	0.40	0.40	0.55	0.22
6281-04	09-16-96	3	2055	200	0.28	0.30	0.52	0.80	0.50
6281-05	09-22-96	7	4195	1700	0.36	0.57	0.64	1.16	0.67
6281-06	09-25-96	2	8765	6500	1.34	1.92	1.56	3.34	1.50
6281-07	10-06-96	7	34894	27000	3.36	6.70	4.05	11.20	5.46
6281-08	10-09-96	6	57910	20000	4.31	8.57	5.25	14.84	7.33
6281-09	10-14-96	6	34100	8100	9.82	14.96	11.04	27.73	17.20
6281-10	10-16-96	6	8566	4700	2.21	4.38	2.78	6.76	3.59
6281-11	10-23-96	0	NOT DETECTED	<100	0.07	0.33	0.36	0.43	0.37
6281-12	10-27-96	5	NOT DETECTED	<100	0.08	0.45	0.37	0.32	0.15

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)	ABBOTT AxSYM Core-M	ABBOTT AxSYM Core (Co/S)	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BEHRINGWERKE Enzygnost HBsAg Monoclonal II	BIOTEST ELISA HBsAg	ABBOTT AxSYM HBsAg	ABBOTT Auszyme MC Dynamic HBsAg
	TEST DATE: 2/4/1998	TEST DATE: 4/7/1998	TEST DATE: 4/7/1998	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 4/5/1998	TEST DATE: 4/6/1998	TEST DATE: 3/19/1998	TEST DATE: 3/31/1998
6281-01	0.117	0.22	0.51	0.77	0.52	0.42	0.31	0.57	0.30
6281-02	0.128	0.19	0.49	0.65	0.51	0.37	0.40	0.59	0.26
6281-03	0.138	0.19	0.52	0.64	0.53	0.40	0.39	0.55	0.20
6281-04	0.245	0.21	0.51	1.34	0.88	0.49	0.37	0.63	0.59
6281-05	0.277	0.22	0.51	1.62	1.16	0.66	0.49	0.81	1.35
6281-06	0.957	0.20	0.49	5.81	3.23	1.29	1.11	1.63	4.12
6281-07	1.096	0.20	0.53	19.25	10.96	2.54	3.02	3.50	15.16
6281-08	1.670	0.20	0.51	26.81	15.94	3.48	5.06	5.11	16.29
6281-09	3.287	0.57	1.41	46.47	23.96	6.94	9.22	9.53	28.78
6281-10	0.840	1.62	2.68	14.28	7.28	1.86	2.09	2.54	10.55
6281-11	0.117	2.47	4.96	0.60	0.53	0.40	0.36	0.55	0.33
6281-12	0.138	2.30	4.35	0.95	0.50	0.44	0.25	0.58	0.03

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6281 Rev05
Effective Date: 11/10/2025



HBV Seroconversion Panel Donor No. 62433 Part Number: HBV6281

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg	SORIN ETI-MAK 3 HBsAg	SORIN (INCSTAR) HBsAg	SORIN ETI-Core IGMK2	SORIN ETI-AB CoreK2 (Co/S)	ORTHO VITROS ECI HBsAg	ORTHO VITROS ECI HBeAg	ABBOTT HBeAg (EIA)	ORTHO VITROS ECI ANTI-HBe
	TEST DATE: 4/30/1998	TEST DATE: 5/5/1998	TEST DATE: UNK	TEST DATE: 5/7/1998	TEST DATE: 5/7/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/12/1998	TEST DATE: UNK
6281-01	0.10	0.21	0.157	0.05	0.16	0.26	0.23	0.192	0.15
6281-02	0.10	0.22	0.196	0.09	0.16	0.23	0.23	0.262	0.17
6281-03	0.27	0.18	0.157	0.11	0.16	0.25	0.23	0.215	0.15
6281-04	0.30	0.31	0.196	0.07	0.18	0.5	0.25	0.177	0.16
6281-05	0.40	0.47	0.333	0.05	0.19	0.56	0.24	0.223	0.15
6281-06	1.11	1.08	0.314	0.06	0.18	2.4	0.26	0.215	0.14
6281-07	5.22	2.21	0.627	0.01	0.19	8.54	0.63	0.323	0.15
6281-08	6.71	4.40	0.902	0.00	0.19	13.4	0.77	0.315	0.14
6281-09	11.12	7.52	1.725	0.82	1.67	27.4	0.74	0.623	0.2
6281-10	2.93	1.70	0.471	3.73	3.11	6.47	0.45	1.415	0.35
6281-11	0.17	0.24	0.144	12.29	4.93	0.23	0.25	0.477	0.45
6281-12	0.10	0.20	0.200	11.71	5.42	0.22	0.23	0.208	0.36

PANEL MEMBER	ORTHO VITROS ECI ANTI-HBc TOTAL	ABBOTT ANTI-HBc TOTAL (EIA)	ABBOTT ANTI-HBc IgM	ORTHO VITROS ECI ANTI-HBc IgM	SORIN ETI-AB AUK3 ANTI-HBs	ORTHO VITROS ECI ANTI-HBs QUANT	ABBOTT IMx AUSAB ANTI-HBs	ORTHO VITROS ECI HBeAG
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/4/1998	TEST DATE: UNK	TEST DATE: 5/4/1998	TEST DATE: UNK
6281-01	3.02	<0.116	0.055	0.01	0.08	1.3	0.41	0.23
6281-02	3.24	0.426	0.055	0.01	0.07	1.7	0.40	0.23
6281-03	3.38	0.306	0.049	0.01	0.12	2.4	0.42	0.23
6281-04	3.29	0.316	0.049	0.01	0.11	2.5	0.44	0.25
6281-05	3.11	0.304	0.053	0.01	0.12	1.9	0.40	0.24
6281-06	3.27	0.296	0.049	0.01	0.09	2	0.44	0.26
6281-07	3.28	<0.116	0.060	0.01	0.23	1.6	0.83	0.63
6281-08	3.27	<0.116	0.049	0.01	0.26	2.7	0.65	0.77
6281-09	0.56	0.378	0.717	0.3	0.21	1.4	0.53	0.74
6281-10	0.26	0.789	2.68	1.33	0.24	4.1	0.58	0.45
6281-11	0.11	3.03	>3.77	2.48	1.03	3.7	0.81	0.25
6281-12	0.18	NOT TESTED	>3.77	1.99	6.08	28.8	9.99	0.23

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6281 Rev05
Effective Date: 11/10/2025