

INFORMATION FOR USE (IFU)

MDXM0017

Document #: MDX-SAL-IFU-021

Revision No: 000

Effective Date: 10-21-2025

Lithium Heparin Anticoagulant

Description: Lithium Heparin Anticoagulant solution is used for IRB-Approved research studies to support diagnostic claims for Lithium Heparin blood tubes.

Intended Use: Lithium Heparin Anticoagulant is added to a whole blood sample to prevent clotting. Add 1mL of the Lithium Heparin Anticoagulant solution per 100mL of whole blood and mix by rocking. This will mimic the same concentration of Lithium Heparin in a blood collection tube.

Product Specifications:

MRNDx Part Number	Volume	Storage Conditions	Stability
MDXM0017-10	10mL	15°C to 30°C	12 months
MDXM0017-50	50mL	15°C to 30°C	12 months
MDXM0017-250	250mL	15°C to 30°C	12 months

For Research Use Only – This product is indented for research and development use and should not be used for clinical diagnostic purposes.

IMPORTANT: Biosafety Level 1: The materials used to formulate this product contain no infectious material that pose hazard to laboratory personnel and the environment. These products are NOT intended for use in the manufacturing and/or processing of injectable products subject to licensure under section 351 of the Public Health Service Act or for any other product intended for administration to humans.

This product was manufactured in a facility under a Quality Management System certified by ISO 13485:2016 for the manufacturing of medical devices.