



INFORMATION FOR USE (IFU)

MDXR0037

Document #: MDX-SAL-IFU-026

Revision No: 001

Effective Date: 12-5-2025

K2EDTA Anticoagulant

Description: K2EDTA Anticoagulant solution is used for IRB-Approved research studies to support diagnostic claims for K2EDTA blood tubes.

Intended Use: K2EDTA Anticoagulant is added to a whole blood sample to prevent clotting. Add 1mL of the K2EDTA Anticoagulant solution per 100mL of whole blood and mix by rocking. This will mimic the same concentration of K2EDTA in a blood collection tube.

Product Specifications:

MRNDx Part Number	Volume	Storage Conditions	Stability
MDXR0037-10	10mL	15°C to 30°C	12 months
MDXR0037-50	50mL	15°C to 30°C	12 months
MDXR0037-250	250mL	15°C to 30°C	12 months

For Research Use Only – This product is intended for research and development use and should not be used for clinical diagnostic purposes.

IMPORTANT: Biosafety Level 1: The materials used to formulate this product contain no infectious material that pose hazard to laboratory personnel and the environment. These products are NOT intended for use in the manufacturing and/or processing of injectable products subject to licensure under section 351 of the Public Health Service Act or for any other product intended for administration to humans.

This product was manufactured in a facility under a Quality Management System certified by ISO 13485:2016 for the manufacturing of medical devices.